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Penis Pump Deluxe No. 517810
CUSTOMER INFORMATION
Dear customer,
Congratulations on the purchase of your "Penis Pump Deluxe". You've made a good choice! With this high-quality vacuum virility pump you can train your penis efficiently into "top form" without great effort.
The aim of these instructions is to give you useful advice and tips, so that you have maximum enjoyment and results with your virility pump.

Use:

This virility pump imitates the natural erection process by creating low pressure. The low pressure causes an increase in blood flow and can lead to a stiffness and swelling of your penis. Concentration of the mind, together with the use of the pelvic muscles positively assist this training.

Take your time and be patient. Regular training can soon show lasting results.

Penis pump parts:

Transparent vacuum pipe with 20cm scale, sleeve, pressure regulator and pressure indicator, hose with special suction pump

Instructions:

1. Mount the hose with the pump onto the connector at the top of the vacuum pipe (see photo on packaging). **The penis pump is now ready for use.**
2. Apply a generous amount of lubricant on the inside of the sleeve and on your penis.
- IMPORTANT:** Only use water based lubricants! Oil based lubricants will damage the material.
- IMPORTANT:** The purpose of the sleeve is to make an air tight seal between the body and the vacuum pump. This allows for a high level vacuum. This is why you should use plenty of lubricant (also in the pubic hair – shaving the pubic hair around the base of the penis has an even better effect), in order to reach an optimal seal/air tightness.
3. Insert a non-erect penis (the best is while standing). The vacuum pipe should be firmly pressed against the body.
4. Pump the air from the vacuum pipe.
- IMPORTANT:** If no vacuum occurs, check the seal on the body.
5. Regulate the vacuum using the pressure regulator above the cylinder so that a small amount of air can get in. Continuously repeat this process. Create a vacuum, let air in and pump it out again. The constant change between low and high pressure makes the penis grow and swell. You can watch the erection through the transparent vacuum pipe and read the centimetres from the scale. You can check the low pressure (up to 76 cm Hg) on the pressure indicator.
- IMPORTANT:** Should you experience pain or other physical complaints, discontinue pumping and open the pressure valve.
6. Once the desired erection has been reached, release the pressure using the valve. Now, you can simply slide off the virility pump.

Cleaning and care:

Clean your penis pump before first and after every use. Simply rinse the transparent vacuum pipe, sleeve insert and hose with pump in clean, lukewarm water. Do not totally submerge as moisture can otherwise enter the pressure indicator. Do not put in the dishwasher!
After cleaning, the sleeve insert should be sprinkled with talcum powder to stop the material from becoming brittle and fragile.
To avoid chemical reactions, store your penis pump separately and keep it from other products made of various materials. Please, keep the penis pump clean, cool and dry. Keep out of sunlight!

Safety precautions:

1. Keep out of reach of children
2. Do not use penis pump on diseased or irritated skin or on infected areas.
3. To avoid diseases and infections, do not let other people use the penis pump.
4. Not to be used by people with haemophilia or penis implants.
5. Please note the material information on the packaging in order to avoid allergic reactions e.g. to Latex or similar.
6. If unpleasant skin tension or irritation occurs, discontinue use immediately.
7. This penis pump can produce very strong low pressure which can cause injury. Use at own risk.
8. Do not heat or freeze penis pump.
9. Careful handling and use of the penis pump will protect against damage and accidents. No responsibility will be taken for damage and accidents caused by improper handling and disregard of these safety precautions.

Information about protection of the environment:
Please do not throw away packaging material, recycle instead.

We wish you much fun and success with your penis pump and hope that your expectations will be fulfilled.

Kindest regards
Your erotic specialist
OV-Grosshandel, 24933 Flensburg, Germany

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