
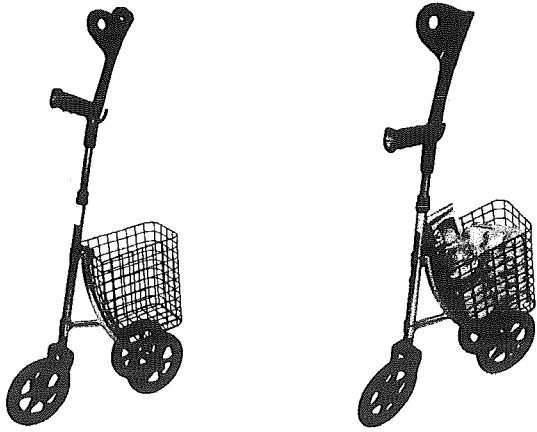
23469


CE Pat.Pending
Made In China

2nd. Assembling instruction

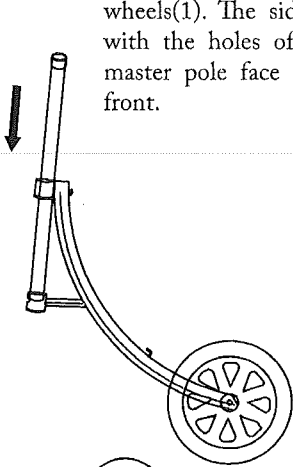
1. Insert master pole(3) into back wheels(1). The side with the holes of master pole face front.

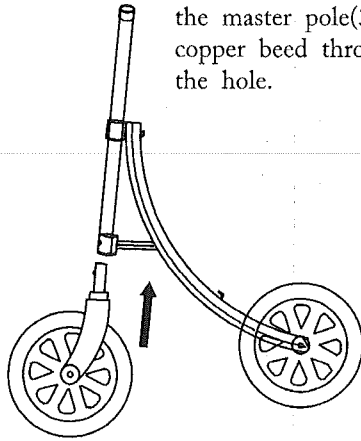
2. Put front wheel(1) into the master pole(3), let the copper bead through out the hole.

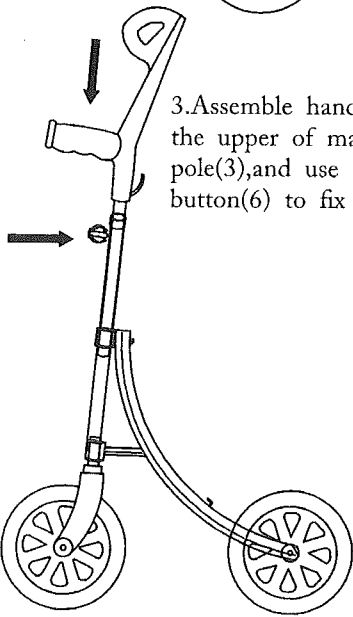3. Assemble handle on the upper of master pole(3), and use "C" button(6) to fix it.

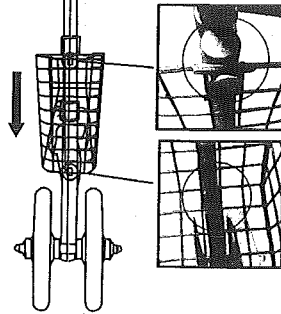
4. Hang the basket on the hook of back wheels(1)

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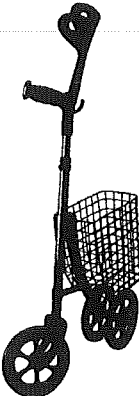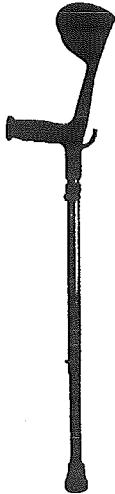
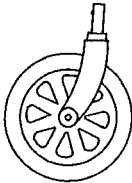
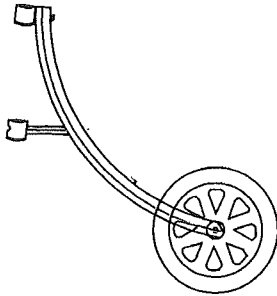
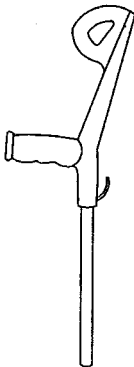
GB Ergo WalkAid is suitable for persons that use a crutch or a walking stick and is good for rehabilitation and exercise. It also works as an ergonomic support aid. WARNING! The product is designed for users with sufficient balance and ability to fend off unexpected movement. If you are unsure whether the device is suitable for you, ask a knowledgeable person. Warning! The product should not be used in slippery conditions. Not to be used by newly operated persons or persons with big balance problem. Max weight: 100 kg. Height adjustment: It is important that the correct height is used. If unsure, consult an expert. Make sure the pin is fully inserted! Warranty: We give two years warranty on your Ergo WalkAid if being used normally. This does not apply to wear items.

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


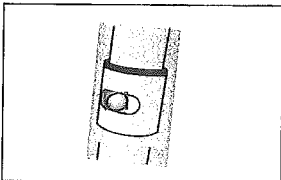
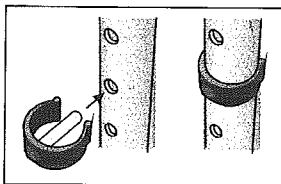
3rd, Revert to crutch instruction.

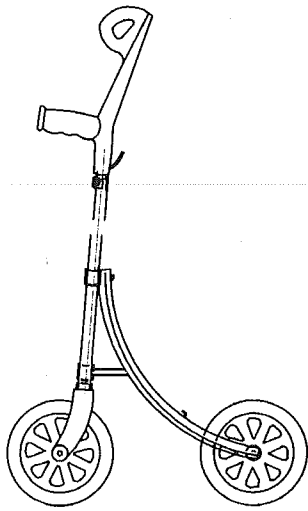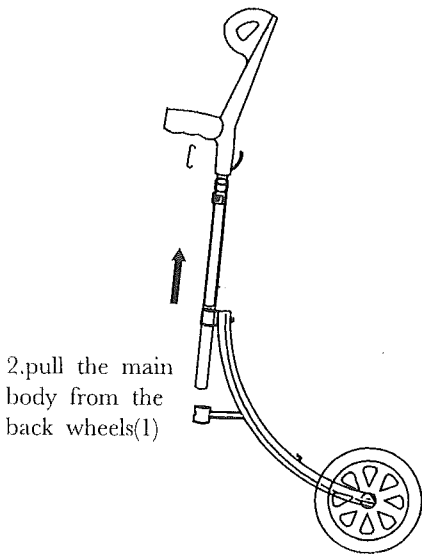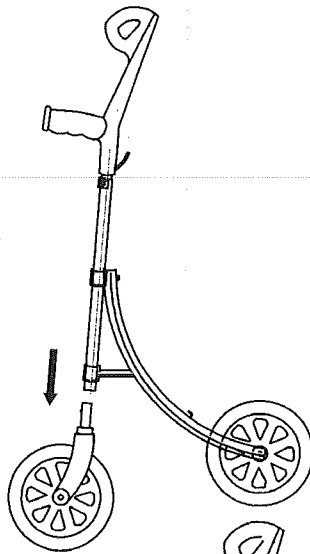
1. Remove the front wheel(2) from the master pole(3)

2. pull the main body from the back wheels(1)

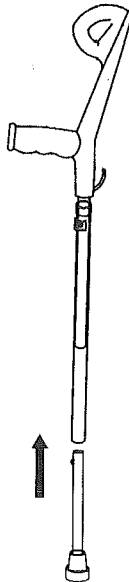3. Assembl the leg tube(7) on the main body.

27469

User's manual for Combi-crutch

1st. Component list:

1. Back wheels 2. Front wheel 3. Master pole

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2nd. Assembling instruction

1. Insert master pole(3) into back wheel(1). The side with the holes of master pole face front.

2. Put front wheel(2) into the master pole(3), let the copper bead through out the hole.

3. Assemble handle on the upper of master pole(3), and use "C" button(6) to fix it.

4. Hang the basket on the hook of back wheel(1).

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3rd. Revert to crutch instruction.

1. Remove the front wheel(2) from the master pole(3).

2. Pull the main body from the back wheel(1).

3. Assemble the leg tube(7) on the main body.

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EN
WARNING! Risk of injury falls. The product is designed for users with sufficient balance and ability to fend off unexpected movement. If you are unsure whether the device is suitable for you, seek a knowledgeable person.
 Warning! Risk of slip injury. The product should not be used in slippery conditions.
 Height adjustment
 It is important that the correct height is used. If unsure, consult an expert. Make sure the pin is fully inserted!

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EN

WARNING! Risk of injury cases. The produkt is designed for users with sufficient balance and ability to fend off unexpected movement. If you are unsure whether the device is suitable for you, ask a knowledgeable person.

Warning! Risk of slip injury. The product should not be used in slippery conditions.

Height adjustment

It is important that the correct height is used. If unsure, consult an expert. Make sure the pin is fully inserted!

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