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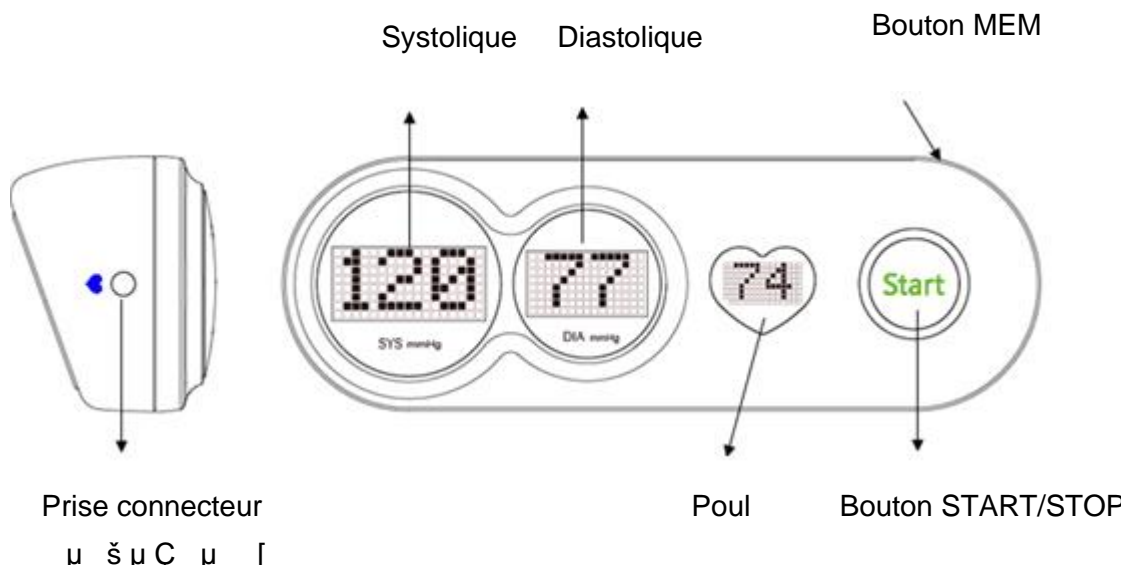
1. Vue générale

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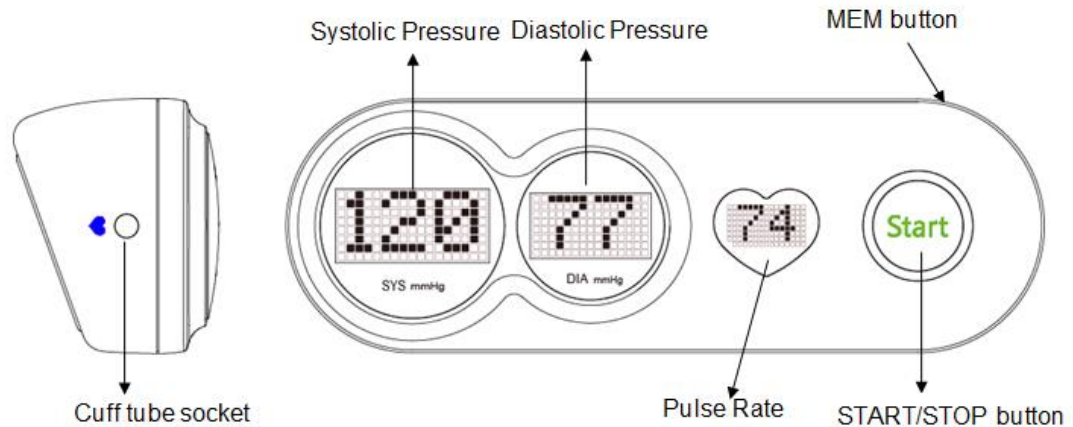
Fully Automatic Arm Cuff Blood Pressure Monitor

1. Overview

! It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.

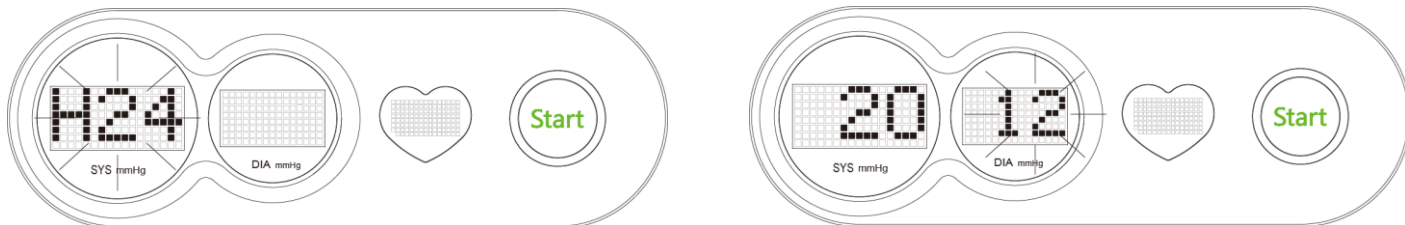
SETUP AND OPERATING PROCEDURES

2. Battery replacement

- a. Open battery cover at the back of the monitor.
- b. Insert the battery with the correct polarity.
- c. Close the battery cover.

3. Clock and Date adjustment


- a. While the monitor is in Off Mode, pressing the MEM button for 3 seconds a beep is heard and the time format will blink at first. See pictures on next page.
 - Press the MEM button again to set the Year, Month, day, hour and minute.
 - Press the MEM button again to set the date.
 - Press the MEM button again to set the time.
 - Press the MEM button again to set the AM/PM.
 - Press the MEM button again to set the number. Keep on pressing the button "MEM", the number will increase fast.
- b. Press the MEM button again to confirm the time and date. The time and date is confirmed.

- c. The monitor will turn off automatically after 1 minute of no operation, with the time and date unchanged.
- d. Once you change the batteries, you should adjust the time and date.

4. CONNECTING THE CUFF TO THE MONITOR

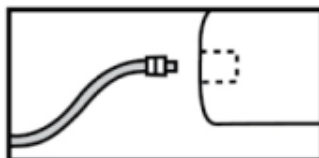
Insert the cuff tubing connector into the socket in the left side of the monitor. Make certain that the connector is completely inserted to avoid air leakage during blood pressure measurements.


- ⚠ Avoid compression or restriction of the connector tubing during measurement which may cause inflation error, or harmful injury due to continuous cuff pressure.

5. Applying the cuff

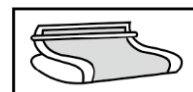- a. Pulling the cuff end through the medial loop (the cuff is packaged this already), turn it outward (away from your body) and tighten and close the elastic fastener.

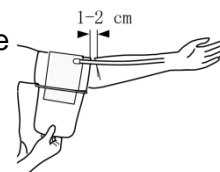
- b. Place the cuff around a bare arm 2 cm above the elbow joint.
- c. While seated, place palm side in front of you on a flat surface such as a desk or table. Position the air tube in the middle of your arm in line with your middle finger.
- d. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger between your arm and the cuff.

6. Body posture during measurement


Sitting Comfortably Measurement

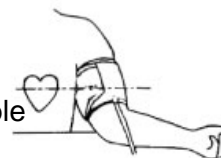- Be seated with your feet flat on the floor, and do not cross your legs.
- Place palm up in front of you on a flat surface such as a desk or table.
- The middle of the cuff should be at the level of the right atrium of the heart.

Lying Down Measurement

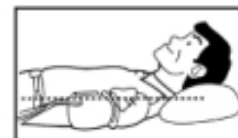
- Lie on your back.
- Place your arm straight along your side with your palm up.
- The cuff should be placed at the same level as your heart.

Once done, press the button Start. The device will automatically straighten your arm, keep calm and relax. Do not talk. Please wait until the screen displays final results. Please find below a template to analyze these results.

7. Display reading

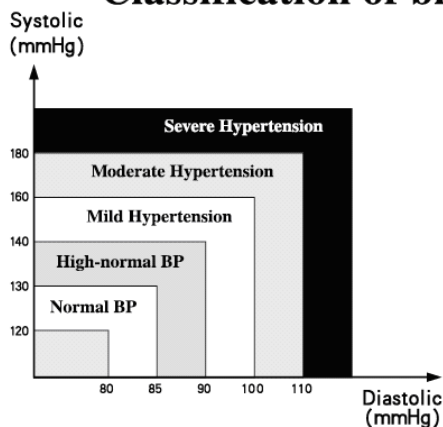Classification of blood pressure for adults

BLOOD PRESSURE CLASSIFICATION	SBP mmHg	DBP mmHg	COLOR INDICATOR
Optimal	<120	<80	GREEN
Normal	120-129	80-84	GREEN
High-Normal	130-139	85-89	GREEN
Grade 1 Hypertension	140-159	90-99	YELLOW
Grade 2 Hypertension	160-179	100-109	RED
Grade 3 Hypertension	≥180	≥110	RED

WHO/ISH Definitions and Classification of Blood Pressure Levels

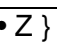
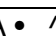
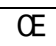
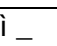
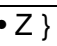
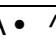
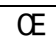
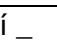
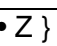
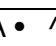
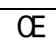
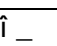
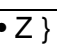
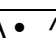
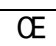
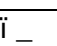
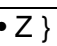
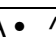
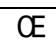
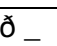
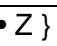
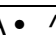
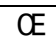
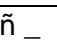
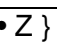
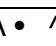
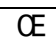
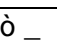
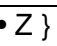
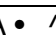
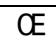
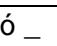
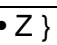
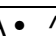
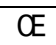
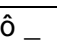
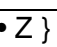
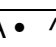
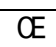
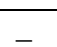
Note: The classification color indicator can be shown different color according to the systolic pressure and diastolic pressure when displaying the measurement.

8. Displaying stored results

Press the **MEMO** button to display the stored results.

If there are results in the memory bank, the LCD will display most recent result including the memory number and blood pressure.

9. Troubleshooting

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD shows low battery symbol 	Low Battery	Change the batteries
>    	Pressure system is unstable before measurement	} v [š u } À try again.
>    	Fail to detect systolic pressure	
>    	Fail to detect diastolic pressure	
>    	Pneumatic system blocked or cuff is too tight during inflation	Apply the cuff correctly and try again
>    	Pneumatic system leakage or cuff is too loose during inflation	
>    	Cuff pressure above 300mmHg	Measure again after five minutes. If the monitor is still abnormal, please contact the local distributor or the factory.
>    	More than 3 minutes with cuff pressure above 15 mmHg	
>    	EEPROM accessing error	
>    	Device parameter checking error	
>    	Pressure sensor parameter error	
No response when you press button or load battery.	Incorrect operation or strong electromagnetic interference.	Take out batteries for five minutes, and then reinstall all batteries.

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD Display shows abnormal result	The cuff position was not correct or it was not properly tightened	Apply the cuff correctly and try again
	Body posture was not correct during testing	Review the instructions and retest.
	Speaking, arm or body movement during testing	Retest when calm and without speaking or moving during the test
	Irregular heartbeat (arrhythmia)	It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.

10. Technical Specifications

Model	KD-5005
Memory volume	120 times with time and date stamp
Power source	batteries 4 x 1.5V  SIZE AAA
Measurement range:	
Cuff pressure	0-300mmHg
Systolic:	60-260mmHg
Diastolic:	40-199mmHg
Pulse rate:	40-180 beats/minute
Accuracy:	
Pressure	
Pulse rate	

MAINTENANCE

1.  Do not drop this monitor or subject it to strong impact.
2.  Avoid high temperature and solarization. Do not immerse the monitor in water as this will result in damage to the monitor.
3. If this monitor is stored near freezing, allow it to acclimate to room temperature before use.
4.  Do not attempt to disassemble this monitor.
5. If you do not use the monitor for a long time, please remove the batteries.
6. Clean the monitor with a dry, soft cloth or a soft cloth squeezed well and moistened with water, diluted disinfectant alcohol, or diluted detergent.

The European directive 2012/19/EC about electronic and electric waste, requires that this appliance be collected separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following tips:
 - Rechargeable battery.

- Leave used batteries and accumulators in adapted containers that you can find in the user manual and the head of the battery.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with the battery.
- Used battery has to be removed.
- Never try to shortcircuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.

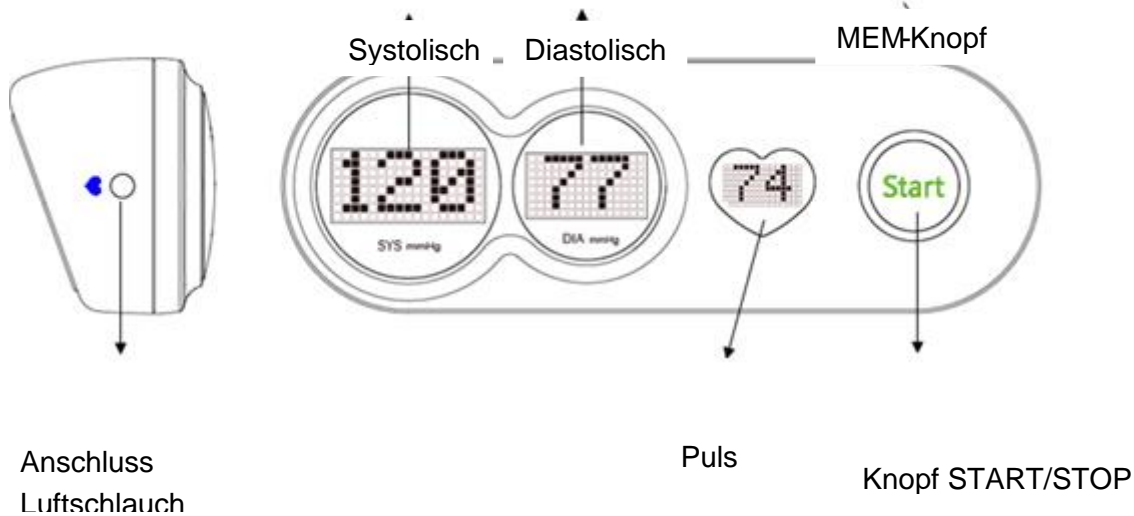
Imported by CAPTELEC, 24 RUE DE MARCQ, 59290 WASQUEHAL FRANCE

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