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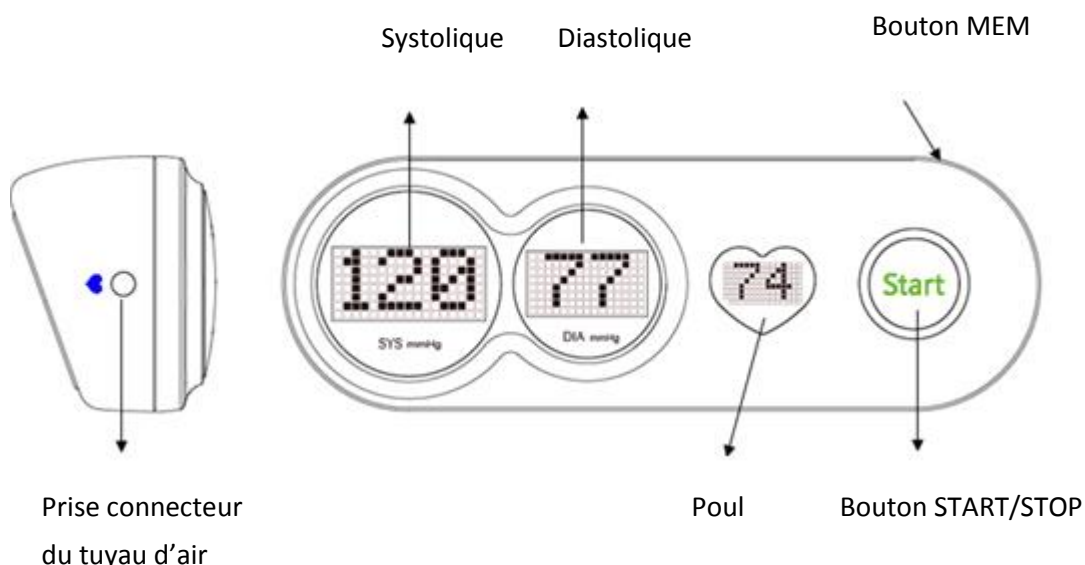
1. Vue générale

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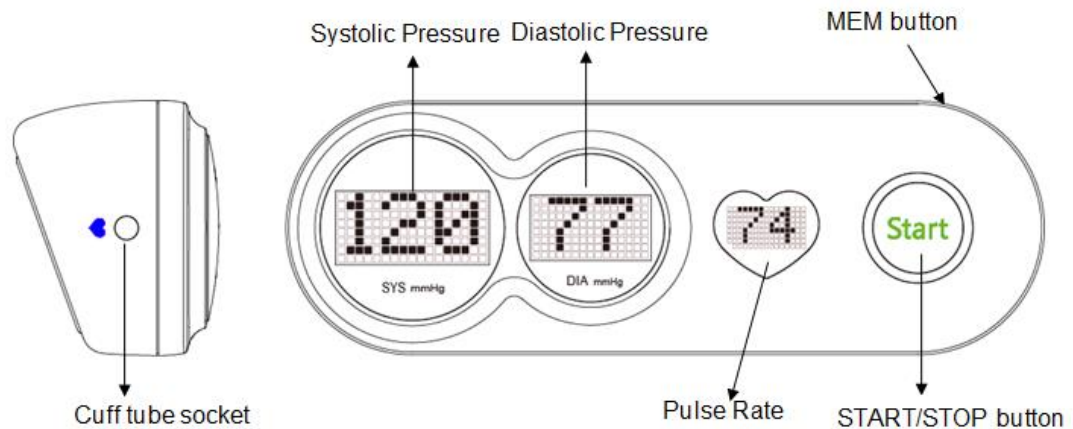
Fully Automatic Arm Cuff Blood Pressure Monitor

1. Overview

⚠ It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.

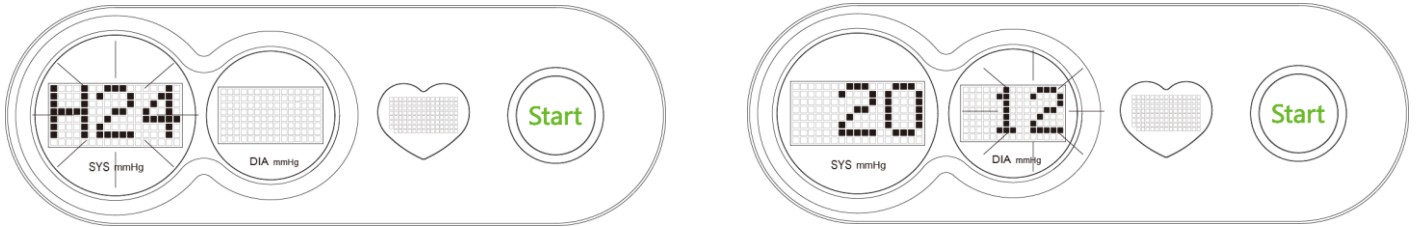
SETUP AND OPERATING PROCEDURES

2. Battery replacement

- a. Open battery cover at the back of the monitor.
- b. Load four "AAA" size batteries. Please pay attention to polarity.
- c. Close the battery cover.

3. Clock and Date adjustment


- a. While the monitor is in Off Mode, pressing the "MEM" button for 3 seconds, a beep is heard and the time format will blink at first. See pictures on next page. Press the button "START/STOP". The sequence is: Year, month, day, hour and minute. While the number is blinking, press the button "MEM" to increase the number. Keep on pressing the button "MEM", the number will increase fast.
- b. You can turn off the monitor by pressing "START/STOP" button when the minute is blinking, then the time and date is confirmed.

- c. The monitor will turn off automatically after 1 minute of no operation, with the time and date unchanged.
- d. Once you change the batteries, you should readjust the time and date.

4. CONNECTING THE CUFF TO THE MONITOR


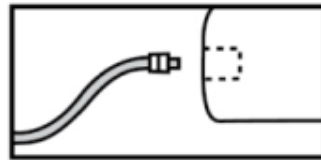
Insert the cuff tubing connector into the socket in the left side of the monitor. Make certain that the connector is completely inserted to avoid air leakage during blood pressure measurements.

- ⚠️ Avoid compression or restriction of the connection tubing during measurement, which may cause inflation error, or harmful injury due to continuous cuff pressure.

5. Applying the cuff

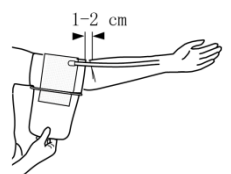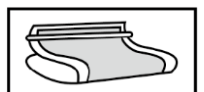- a. Pulling the cuff end through the medal loop (the cuff is packaged like this already), turn it outward (away from your body) and tighten it and close the Velcro fastener.
- b. Place the cuff around a bare arm 1-2cm above the elbow joint.
- c. While seated, place palm upside in front of you on a flat surface such as a desk or table. Position the air tube in the middle of your arm in line with your middle finger.
- d. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger between your arm and the cuff.

6. Body posture during measurement

Sitting Comfortably Measurement

- a. Be seated with your feet flat on the floor, and don't cross your legs.
- b. Place palm upside in front of you on flat surface such as a desk or table.
- c. The middle of cuff should be at the level of the right atrium of the heart

Lying Down Measurement

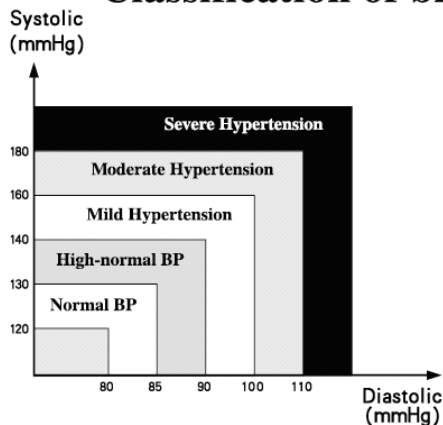
- a. Lie on your back.
- b. Place your arm straight along your side with your palm upside.
- c. The cuff should be placed at the same level as your heart.

Once done, press the button Start. The device will automatically straight your arm, keep calm and relax. Do not talk. Please wait until the screen displays final results. Please find below a template to analyze these results.

7. Display reading

Classification of blood pressure for adults

BLOOD PRESSURE CLASSIFICATION	SBP mmHg	DBP mmHg	COLOR INDICATOR
Optimal	<120	<80	GREEN
Normal	120-129	80-84	GREEN
High-Normal	130-139	85-89	GREEN
Grade 1 Hypertension	140-159	90-99	YELLOW
Grade 2 Hypertension	160-179	100-109	RED
Grade 3 Hypertension	≥180	≥110	RED

WHO/ISH Definitions and Classification of Blood Pressure Levels

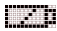
Note: The classification color indicator can be shown different color according to the systolic pressure and diastolic pressure when displaying the measurement.

8. Displaying stored results

After measurement, you can review the results in the memory bank by pressing the “MEM” button. Alternatively, you can press “MEM” button in Off Mode to display the stored results.

If there are results in the memory bank, the LCD will display the most recent result including the memory number and blood pressure.

9. Troubleshooting

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD shows low battery symbol 	Low Battery	Change the batteries
LCD shows “Er 0”	Pressure system is unstable before measurement	Don't move and try again.
LCD shows “Er 1”	Fail to detect systolic pressure	
LCD shows “Er 2”	Fail to detect diastolic pressure	
LCD shows “Er 3”	Pneumatic system blocked or cuff is too tight during inflation	Apply the cuff correctly and try again
LCD shows “Er 4”	Pneumatic system leakage or cuff is too loose during inflation	
LCD shows “Er 5”	Cuff pressure above 300mmHg	Measure again after five minutes. If the monitor is still abnormal, please contact the local distributor or the factory.
LCD shows “Er 6”	More than 3 minutes with cuff pressure above 15 mmHg	
LCD shows “Er 7”	EEPROM accessing error	
LCD shows “Er 8”	Device parameter checking error	
LCD shows “Er A”	Pressure sensor parameter error	
No response when you press button or load battery.	Incorrect operation or strong electromagnetic interference.	Take out batteries for five minutes, and then reinstall all batteries.

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD Display shows abnormal result	The cuff position was not correct or it was not properly tightened	Apply the cuff correctly and try again
	Body posture was not correct during testing	Review the "BODY POSTURE DURING MEASUREMENT" sections of the instructions and re-test.
	Speaking, arm or body movement, angry, excited or nervous during testing	Re-test when calm and without speaking or moving during the test
	Irregular heartbeat (arrhythmia)	It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.

10. Technical Specifications

Model	KD-5005
Memory volume	120 times with time and date stamp
Power source	batteries: 4 ×1.5V  SIZE AAA
Measurement range:	
Cuff pressure	0-300mmHg
Systolic:	60-260mmHg
Diastolic:	40-199mmHg
Pulse rate:	40-180 beats/minute
Accuracy:	
Pressure	
Pulse rate	

MAINTENANCE

1.  Do not drop this monitor or subject it to strong impact.
2.  Avoid high temperature and solarization. Do not immerse the monitor in water as this will result in damage to the monitor.
3. If this monitor is stored near freezing, allow it to acclimate to room temperature before use.
4.  Do not attempt to disassemble this monitor.
5. If you do not use the monitor for a long time, please remove the batteries.
6. Clean the monitor with a dry, soft cloth or a soft cloth squeezed well after moistened with water, diluted disinfectant alcohol, or diluted detergent.

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

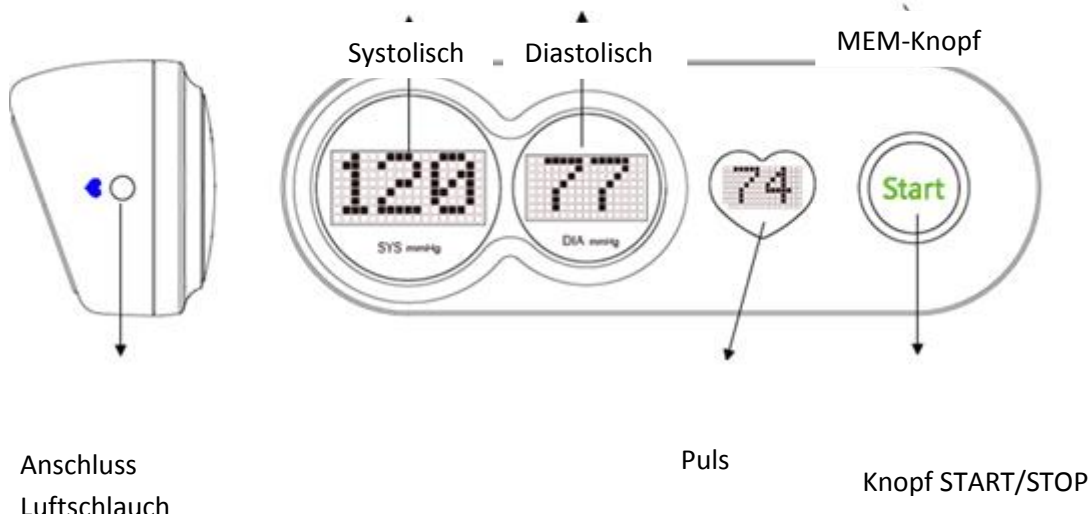
- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.

- Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.

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