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GB Instructions for Use and Safety Information

Heat has always had a pleasant, beneficial effect.

The Wundmed heat belt provides soothing heat for relaxation and wellness in sport, leisure or at work. The heat belt is made of soft, skin friendly material.

The heat pads contain a mixture of iron powder and activated carbon. As soon as this mixture comes into contact with oxygen/air, it warms up and after about 30 minutes produces uniform heat for around 8 hours. The heat pads are inserted in the heat belt, which is then applied and fixed in place at the lumbar spine or hip region.

Usage:

• Do not use

- if skin is pre-damaged or diseased
- in case of swelling, bruises or inflammation
- if the pads are damaged.

• Open the sachet containing the heat pad by hand (do not cut into it) and take out the heat pad.

• The heat pad is activated by the air. Please only open when it is about to be used.

• Place a pad in each of the two pouches provided for this purpose.

• Do not insert more than one pad in each pouch.

• Place the heat belt closely fitted around the body and fix it in place with the Velcro fastener.

• The heat pads warm up slowly and within about 30 minutes start radiating their beneficial warmth, which lasts for up to 8 hours.

• After using the heat pads, take them out of the belt and

dispose off them in the household waste.

• Each pad must be used once only but the heat belt is completely re-usable. Further pads can be purchased separately.

• The heat belt is intended solely for external use in the lumbar or hip region.

Safety information

• With children, the elderly or handicapped, use only under supervision.

• If used improperly, the heat pads can lead to skin irritation even at low temperatures. Please adhere strictly to the instructions for use.

• In rare cases, heat-producing pads can lead to skin irritation!

• Persons with sensitive skin or those who have previously shown allergic reactions to medicinal products or cosmetics, should wear the heat belt over thin clothing and not directly on the skin.

• Do not use in conjunction with creams, body lotions or ointments.

• Never warm up in a microwave oven!

• You should stop using the pads if they become unpleasantly hot or cause a skin rash.

• No tightly fitting clothing, waistbands or other belts should be worn over the heat belt. Please allow air to circulate above the heat belt.

• Please do not lay or sit on the heat belt. Do not lean against hard surfaces!

• Do not use immediately after bathing or showering or whilst sleeping.

• Depending on your skin type - at least 48 hours should elapse between periods of use.

Storage

• Store the heat pads at room temperature (25°C).

• Store heat pads out of the reach of children and domestic pets.

• Protect against dampness

Please adhere strictly to the instructions for use of this heat pad, since WUNDmed GmbH is unable to accept any liability in the event of improper use.

This heat pad is a wellness product and not a medication or medicinal product!

If use for any medicinal purpose is envisaged or intended, the user should consult a doctor.

WUNDmed®

Art.-Nr. 04-031
Wundmed GmbH & Co. KG
91183 Abenberg · Dieselstr. 5
0 91 78 / 99 68 77
www.wundmed.de