### A. Battery Installation

Open the battery door use finger tip and insert 1 x 1.4v zinc air battery (A312 special for mini amplifier) as shown (FIG. 1)


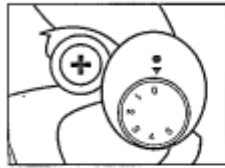
Abb. 1

Press down to shut the battery door of unit.

### B. ON/OFF SWITCH

The battery door can also act as the ON/OFF switch. There are two lines engraved into the battery door. (FIG. 2) When battery door is shut off between the two engraved lines this is ON. When battery door is shut off as first engraved line this is OFF.

Abb. 2

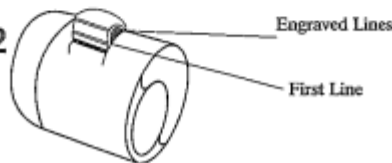3. Use your finger to control and adjust volume at suitable level. (FIG. 5)

Abb. 5

Be careful your hands is cover the MINI AMPLIFIER.  
If cover you will receive high pitch sound.


4. After use turn battery door at OFF position shown as (FIG. 2)

When battery is spend over 24 hours continue sound will be lower. May be you need to adjust at high level or replace new battery to keep unit at your suitable level.

### E. MAINTENANCE WITH ACCESSORIES

1. Use plastic stick end of sharp to open battery door. (FIG. 6)

Abb. 6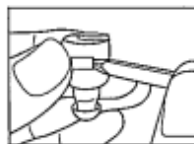

### C. VOLUME CONTROL

The volume increase from clockwise position (FIG. 3)

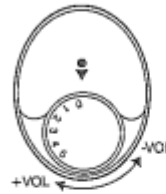
Abb. 3

Users are recommended to switch the volume control from lower level and then adjust to suitable hearing level.

### D. CORRECT WAY TO WEARING AND USE

1. Shut down battery door at ON position. And adjust VOLUME CONTROL in suitable level.
2. Insert unit inside of ear. Ear tip of earphone must be inserted deeply into the ear. (FIG. 4)

make sure ear tip is tight and the tips is pushed fully on the unit.

Abb. 4

Ear Tip of Earphone must be inserted deeply into the ear.

2. Use a soft brush to brush gentle back and across the receiver end remove the ear grease. This prevent ear grease buildup at receiver end (FIG. 7)

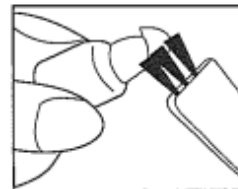
Abb. 7

3. Use plastic with steel to throught pass the receiver end make the ear grease came out. (FIG. 8)

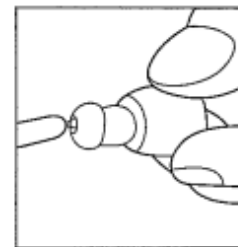
Abb. 8

Never use water for cleaning. Keep unit is dry.

## F. CAUTION

1. The products is not water proof therefore the users must take care to prevent water from entering the unit. Should water enter the MINI AMPLIFIER remove the battery and place the unit in a warm place to dry.
2. It is important to increase the volume control very gradually to avoid sudden increase in sound.
3. This product is not a medical device.

### Technical specific

BATTERY LIFE TIME	20HRS WORKING HOUR (A312 1PCS)
OPERATING CURRENT	5-8mA
POWER CONSUMPTION	10mW
FREQUENCY RESPONSE	50~12K+/-2.5dB
SIGNAL TO NOISE RATIO	>35dB
SENSITIVE	-8dB
WEIGHT (BATTERY INCLUDED)	3.2g

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