


TCL 410

TCL 400

TCL 400 / TCL 410

- Bedienungsanleitung (DE)**
- Operating Instructions (UK)**
- Mode d'emploi (FR)**
- Bedieningshandleiding (NL)**
- Istruzioni d'uso (IT)**
- Instrucciones de uso (ES)**
- Bruksanvisning (SE)**
- Manual de instruções (PT)**
- Bruksanvisning (NO)**

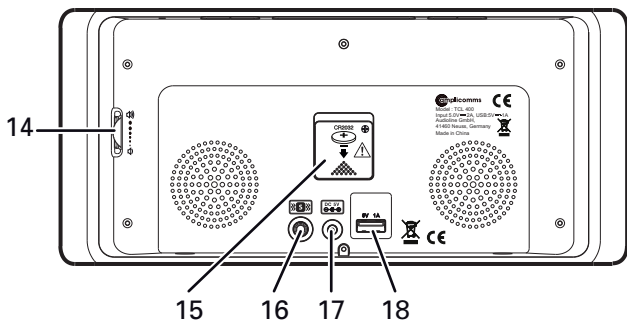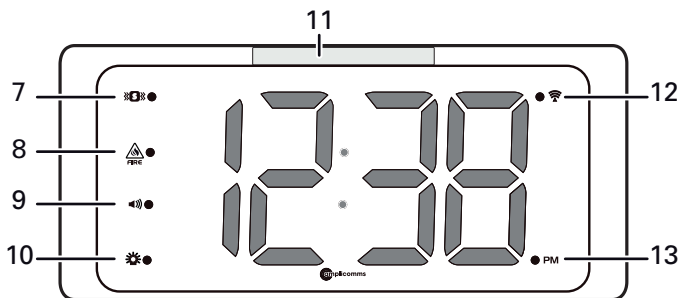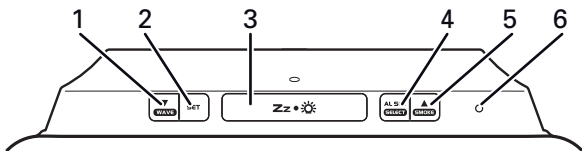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www.amplicomms.com.

- 1 WAVE/▼ button
- 2 SET button
- 3 Snooze/Dimmer button
- 4 AL SET/SELECT button
- 5 SMOKE/▲ button
- 6 Microphone
- 7 LED "Shaker alarm on"
- 8 LED "Smoke detection alarm on"
- 9 LED "Sound alarm on"
- 10 LED "Flash/strobe light alarm on"
- 11 Flash/strobe light
- 12 LED "Radio controlled signal reception"
- 13 LED "PM"
- 14 Alarm volume loudness
- 15 Battery compartment (backup battery)
- 16 Shaker jack
- 17 Power supply
- 18 USB charging port (for charging mobile phone etc.)

Box contents

- Alarm clock
- Power supply 5 V, 2 A
- User guide
- Vibration pad / shaker (TCL410 only)

Installation

Attention: Only use the power adapter plug contained in the material supplied since other power adapter plugs could damage the product. Ensure access to the power adapter plug is not obstructed by furniture or such. The socket-outlet shall be installed near the equipment and shall be easily accessible.

Back-up battery

With the back-up battery (CR2032 Lithium battery) installed, the clock will keep the time and alarm settings in the event of a power cut or the DC adaptor being unplugged. During a power cut, the clock will not display the time, and the alarm function will not work. If no battery is installed and a power cut occurs, then the clock will lose its time and alarm setting. This is important as if there is a power cut in the night then the alarm will not be set correctly to wake you up in the morning at the required time.

Check the battery condition after one year, replace an old battery. A leaking battery can destroy your clock.

For replacing the battery, place your unit face down on a flat surface. Use screw driver to open the backup battery compartment door located on the back of the unit. Slide and remove the battery cover. Insert one new CR2032 lithium battery into the battery compartment with the “+” side facing up as indicated. Close the battery compartment door and tighten the battery door screw.

CAUTION: Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type.

WARNING: Battery shall not be exposed to excessive heat such as sunshine, fire or the like.

WARNING: Do not ingest the battery, Chemical Burn Hazard
This product contains a coin/button cell battery. If the coin/button cell battery is swallowed, it can cause severe internal burns in just 2 hours and can lead to death. Keep new and used batteries away from children. If the battery compartment does not close securely, stop using the product and keep it away from children. If you think batteries might have been swallowed or placed inside any part of the body, seek immediate medical attention.

Environmental reception effects

Your radio controlled clock obtains the accurate time with wireless technology. Same as all wireless devices, the receiving ability maybe affected by, but not limited to, the following circumstances:

- Long transmitting distance
- Nearby mountains and valleys
- Among tall buildings
- Near railway, high voltage cable etc.
- Near freeway, airport, etc
- Near construction site
- Inside concrete buildings
- Near electrical appliances

- Near computers and TV's
- Inside moving vehicles.

Place your clock at a location with optimal signal, i.e. close to a window and away from metal surfaces or electrical appliances.

Radio controlled time signal reception

After your clock is powered up, it starts to receive radio controlled time signal with the time digits at low brightness. The  LED flashes.

- Receiving signal ( LED flashing)
- Successful reception ( LED becomes static)
- Failed reception ( LED disappear).

Reception mode and signal strength

During reception, press **WAVE/▼** once will display reception mode and signal strength indication:

- When strong signal is detected, display shows with 3 bars .
- When weak or no signal is detected, display shows  or .
- During reception, the signal strength may move from 1 bar to 2 bars to 3 bars. This is normal since the clock is detecting radio controlled time signal and other signals in the air at the same time.
- Press **WAVE/▼** once will return to the time display with  LED flashing.

Automatic reception and manual reception

Automatic reception

Your clock starts automatic reception everyday at 1:00, 2:00, and 3:00. If it fails to receive the radio controlled time signal at 3:00, it will start reception at 4:00. If it fails again, it will start reception at 5:00. If it fails again, it will start automatic reception at 1:00 again in the next day.

Manual reception

Simply press and hold **WAVE/▼**, your clock will give a short beep and starts manual reception. Press and hold **WAVE/▼** to stop radio controlled time reception.

When your clock is receiving the radio controlled time signal, the display will turn dim. It will resume to the brightness level which you originally selected after the radio controlled time reception is completed.

During reception, all buttons do not function.

To set time and the snooze duration

(U.K. version: default time at 12:00 AM and 12Hr time format. No time zone selected.)

(German version : default time at 0:00 and 24Hr time format)

1. Press **SET** once, "00" flash, press ▲/▼ to set the time zone (for german version only).
"00" = GMT +1 hour (e.g. Germany)
"01" = GMT +2 hours (e.g. Finland)
"-01" = GMT 00 hour (e.g. U.K.)
2. Press **SET** again, the hour digits flash. Press ▲ or ▼ to set the time in hours (hold ▲/▼ to advance rapidly).
3. Press **SET** again, the minute digits "00" flash. Press ▲ or ▼ to set the time in minutes (hold ▲/▼ to advance rapidly).
4. Press **SET** again, the display shows time format and flashes. Press ▲ or ▼ to select "12HR" or "24Hr" time format.
If "12Hr" is selected, the **PM** LED will appear on the lower right to indicate the afternoon time; there is no AM indicator.
5. Press **SET** again, the display shows "05" and flashes. Press ▲ or ▼ to select your snooze time from 5 to 60 minutes.
6. Press **SET** again or if no key pressed for approx. 10 seconds to exit the setting mode.

To set the alarm

1. Press and hold **AL SET/SELECT**, the **»E«** LED and the alarm's hour digits flash. Press **▲** or **▼** to set the alarm in hours (hold **▲/▼** to advance rapidly).
2. Press **AL SET/SELECT** once again, the alarm's minute digits flash. Press **▲** or **▼** to set the alarm in minutes (hold **▲/▼** to advance rapidly).
3. Press **AL SET/SELECT** again, LED displays "**HI**" (vibration at HI level), press **▲** or **▼** to select "**LO**" (vibration at Lo level) if need.
4. Press **AL SET/SELECT** once again, LED displays "**F-1**" and flash, press **▲** or **▼** to select the alarm tone (frequency).
 - F-1 (low tone, 800 Hz)
 - F-2 (mid tone, 1 KHz)
 - F-3 (high tone, 1.5 KHz)
5. Press **AL SET/SELECT** once again or no button around 10 seconds to exit the alarm setting mode.

To select the alarm mode

1. At normal time display mode, press **AL SET/SELECT** one at a time to activate the alarm(s), the corresponding alarm indicator(s) on the left side of the time display will be on:
 - ⌘: only shaker alarm is activated
 - 🔊 ⌘: both sound and shaker alarm are activated
 - ☀️ ⌘: both strobe light and shaker alarm are activated
 - ☀️ 🔊 ⌘: strobe light, sound and shaker alarm are activated
2. Press **AL SET/SELECT** once more to turn off all the alarms, all ⌘, 🔊 and ☀️ alarm indicators are turned off.

GB


Adjust your alarm volume (buzzer)

You can turn the volume dial at the right side of your clock to adjust the alarm volume.

Use smoke alarm detection

1. Press and hold **SMOKE/▲** button, the  LED is turned on and smoke alarm detection is activated. Once the microphone of your clock receives the alert sound of a smoke alarm, the  LED flashes and your clock will alert you by:
 - Flashing "AL" display
 - Alarm sound (depends on the alarm volume setting)
 - HI brightness of flashing LED
 - Shaker vibration.
2. To stop the above 1-4 alarms, press **SMOKE/▲** button once.  LED remains static.

Caution: the smoke alarm detection is effective 2.5 to 3 meters down from the smoke alarm device. It does not cover a larger area nor being effective across the room.

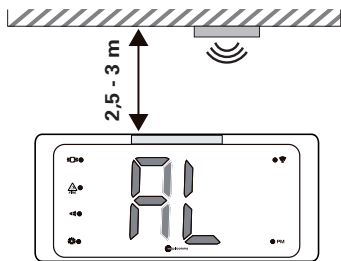Check from time to time if there is any debris in the microphone opening. Clean it with a soft brush if need.

Checking the smoke alarm detection


To check the function: please press the "Test-button" on your smoke detector and check if TCL 400/410 is within the range of your existing smoke detector. If the TCL 400/410 detects the alarm from the smoke detector the alarm is given by:

- Flashing "AL" display
- Alarm sound (depends on the alarm volume setting)
- HI brightness of flashing LED
- Shaker vibration.


To stop and reset the alarm

When alarm is sounding, the corresponding alarm indicator flashes. Press **AL SET/SELECT** once to stop the alarm and reset it to come on the following day. After that, the corresponding alarm indicator stays on the display.

To use snooze

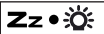When alarm is sounding, press  once, the alarm will be stopped temporarily and the corresponding alarm indicator keeps flashing. Alarm will come on again after the set snooze duration is over.

To use the HI-MED-LOW dimmer

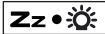There is a 3-level brightness control for the LED display. Press  to adjust the brightness (HI / MED / LO) of the LED display. This can only be done when the unit is not sounding.

To charge your mobile device with the USB port

Connect the charging cable (not included) of your mobile device to the A-type USB port at the back of the unit, then connect the other end to your mobile device. The clock will charge the mobile device automatically.

Remarks

- Make sure the original cable of your mobile device is used for charging
- Make sure the original adaptor enclosed with your clock radio (5V 2A) is being used to power your clock. Using other adaptors may not charge your phone.
- iPad and notepad charging is not applicable.
- No USB charging provided during radio controlled time reception or during the alarms are sounding / active.

Technical data / features

Functions: Clock, Alarm clock

Alarm modes: Buzzer, Vibration, Strobe light

Operating voltage:

Power supply Input 100~240 V AC, Output 5 V DC / 2 A.

Back-up battery (optional, not included):

CR2031 lithium battery, change once a year.

Specifications

Alarm duration	1 hour
Snooze duration	5 to 60 minutes
Default settings:	
Time format	24 hour (German version), 12 hour (U.K. version)
Time	0:00 (German version), 12:00 (U.K. version)
Alarm	6:00 (AM)
Alarm and shaker level	HI
Alarm sound frequency	F-1
Smoke alarm	OFF
Snooze duration	05 (5 minutes)

Safety Information

- Read this manual carefully and keep it for future reference.
- Never use electrical equipment near water, e.g. close to bathtubs, sinks, swimming pools, sinks, damp cellars or similar places.
- Place your device on a stable surface, away from sources of direct sunlight or excessive heat or moisture.
- Leave sufficient space around the device. Please check that the air circulates freely around the device.
- Make sure that the ventilation openings of the device are not blocked. Do not place the device on beds, sofas, carpets, or similar surfaces where the ventilation openings can be easily clogged. Do not place the device in enclosed areas, such as in bookcases or in built-in cabinets. Remove any objects, such as tablecloths, curtains, or newspapers that can block the ventilation openings of the device.
- Protect the device from dripping or spraying water. Do not place any containers with liquid, such as vases, on or near the device. The container may tip over and the liquid may limit the electrical safety.
- Do not operate the device and power supply in the open-air or in very humid areas.
- Only connect the device to a properly installed socket with a mains voltage that corresponds to the voltage on the power supply.

- Unplug the power supply from the socket:
 - during storms
 - before you clean the connected device
 - if faults arise during operation
 - if you do not use the device for a longer period of time.Always unplug the power supply, not the connection cable.
- Do not open or remove parts of the casing under any circumstances. Inside the casing are live parts that can cause electric shock if touched.
- Do not insert any objects into the openings on the device. They might come into contact with live parts.
- The connection cables must not be bent or crushed. Keep the connection cable away from hot surfaces and sharp edges.
- Check the devices and the power supply for damage regularly. Do not turn on any device if one of its components, the power supply or the connection cable has visible damage or a device has fallen.
- Protect your furniture when placing your units on a natural wood and lacquered finish by using a cloth or protective material between it and the furniture.

Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The symbol on this product indicates that electrical and electronic apparatus must be disposed of separately from domestic waste at suitable collection points provided by the public waste authorities. Packaging materials must be disposed of according to local regulations.

Maintenance

- Clean your unit with a soft cloth moistened only with mild soap and water. Stronger agents such as Benzene, thinner or similar materials can damage the surface of the unit. Make sure the unit is unplugged before cleaning.
- If the unit is not to be used for a prolonged period, such as a month or longer, remove the battery to prevent possible corrosion. Should the battery compartment become corroded or dirty, clean the compartment thoroughly and replace the battery.
- Use a brush to gently clear any debris from the microphone from time to time. The brush should have soft, clean, dry bristles.

Troubleshooting

If your clock displays irrelevant time or does not function properly, which may be caused by electro-static discharge or other interference, unplug the DC jack to disconnect the AC power (and remove the backup battery), leave it for few minutes and then plug in the adaptor to repower your clock.

The clock will be reset to default settings and you need to set it again.

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The period of guarantee is 24 months from the date of purchase. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. All deficiencies resulting from material of production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device.

Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

Helpline

If you need some help and you're in the UK, please call us on 0844 800 6503 (using an UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays.

If you've a claim on your 2-year guarantee, please contact your supplier.

Declaration of conformity

This device fulfils the requirements stipulated in the EU directive: 1999/5/EC.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website

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Audioline GmbH