Gebrauchsanleitung | Instruction leaflet | Instruction d'utilisation  
Instrucciones de empleo | Istruzioni per l'uso | Gebruiksaanwijzing  
Руководство по эксплуатации | Instrukcja użytkowania | Kullanım kılavuzu

## promed BDP-200

Oberarm-Blutdruckmessgerät  
Upper Arm Blood Pressure Monitor  
Tensiomètre de bras  
Misuratore di pressione arteriosa da braccio  
Medidor de la tensión arterial para el brazo  
Bovenarmbloeddrukmeter

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AR

## TABLE OF CONTENTS

COMMISSIONING	p. 20
SCOPE OF DELIVERY / ACCESSORIES	p. 21
DESCRIPTION OF THE DEVICE	p. 22
PROPER USE	p. 22
MEDICAL NOTE	p. 22
SAFETY INSTRUCTIONS	p. 23/24
INTRODUCTION	p. 25/26
OPERATING THE DEVICE	p. 27-30
TROUBLESHOOTING	p. 31
CARE, MAINTENANCE AND STORAGE	p. 32
DISPOSAL	p. 32/33
WARRANTY TERMS	p. 33/34
EXPLANATION OF SYMBOLS	p. 146/147
CE DIRECTIVES	p. 148/149
TECHNICAL DATA	p. 150/151
TRANSPORTATION / STORAGE CONDITIONS	p. 154/155
TYPE LABEL / CERTIFICATION NUMBER	p. 151
WARRANTY CARD	p. 156/157
MANUFACTURER AND ADDITIONAL INFORMATION	p. 151

DE

EN

FR

IT

ES

NL

RU


PL

TR

AR

## BEFORE COMMISSIONING

### About this device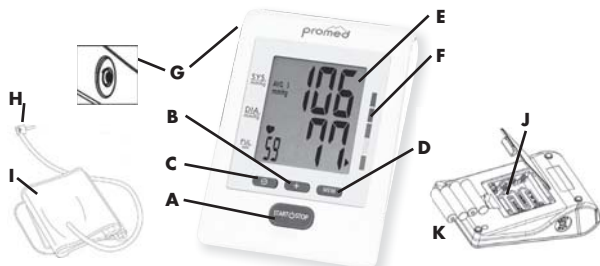

- A. Start / Stop button
- B. Set button
- C. Mode button
- D. Memory button
- E. LCD display
- F. WHO scale

- G. Tube connection
- H. Tube connector
- I. Arm cuff
- J. Battery compartment
- K. Batteries

DE

EN

FR

IT

ES

NL

RU

PL


TR

AR

### **ATTENTION!**

The replacement of a component which differs from the supplied component may result in a measurement error.

### LCD display

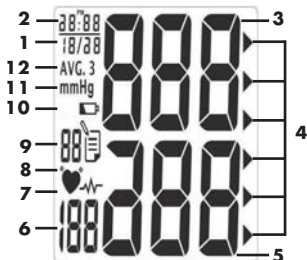- 1. Date display
- 2. Time display
- 3. Systolic blood pressure
- 4. Blood pressure analysis display
- 5. Diastolic blood pressure
- 6. Pulse rate display

- 7. Display for irregular heartbeat
- 8. Display for the pulse rate
- 9. Memory display
- 10. Battery status display
- 11. Measurement unit set
- 12. Display for average values

## BEFORE COMMISSIONING

### Explanation of Display Symbols

SYMBOL	Meaning
 Battery status display	This symbol appears when the battery is empty or the polarity is reversed. When this symbol appears, we recommend you to replace all batteries by new ones and to guarantee that the +/- poles are correctly positioned.
 Display for the pulse rate	When a pulse is measured, the symbol flashes with each pulse beat.
 Display for irregular heartbeat	This symbol appears for 1 minute when the monitor detects an irregular heart rhythm.
 Memory display	With the adjacent number, this symbol shows the selected memory.
 Blood pressure analysis display	The arrowhead indicates the colour index of the WHO scale which matches to your measured value.
<b>AVG. 3</b> Average of the last 3 measurements	This symbol shows that the average value of the last 3 measurements is displayed.

### SCOPE OF DELIVERY / ACCESSORIES

The following components are included in the scope of delivery of your **Promed BDP-200** blood pressure monitor:

- 1x blood pressure monitor
- 1x arm cuff, size M (arm circumference of 23-33cm)
- 1x tube connector
- 4x batteries, 1.5VDC AAA
- 1x storage bag
- 1x Instructions for use
- 1x blood pressure pass

You can buy the following spare parts at Promed:

- arm cuff, size M (arm circumference of 23-33cm), part no.: 402041
- tube connector, part no.: 402042
- batteries, 4x 1.5VDC AAA, part no.: 359994
- storage bag, part no.: 964101
- blood pressure pass, part no.: 984006

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## DESCRIPTION OF THE DEVICE Please keep the instructions carefully!

Please read the user manual thoroughly before using the device for the first time. Many thanks for purchasing the **Promed BDP-200** blood pressure meter. You have acquired a high-quality medical product for the personal checking of health.

**Promed** is a leading company with decades of experience in personal hygiene, wellness and health areas.

The **Promed BDP-200** unit has been designed and manufactured in accordance with the Medical Device Directives 93/42/EEC to guarantee quality during use. The device may be used once this user manual has been read.

We, as the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this user manual. We wish you a lot of enjoyment with your new **Promed BDP-200**. In the following we would like to familiarise you with the **Promed BDP-200** blood pressure meter.

Please read the user manual thoroughly before using the device for the first time.

You can receive professional advice wherever **Promed** products are sold or get in touch with us if you have any questions. We can provide you with the name of the representative responsible for you.

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Fax: +49 (0) 8821/9621-21, E-Mail: info@promed.de

EN

Further informations can be found on our homepage: [www.promed.de](http://www.promed.de)

FR

### **PROPER USE**

IT

**Promed BDP-200** is a blood pressure monitor which automatically measures the systolic and diastolic blood pressure using the oscillometric measuring method. Furthermore, it measures the pulse and displays an irregular heartbeat, where appropriate. It is only intended for the application to human beings. **Promed BDP-200** has been developed for the use at home and should only be used by adults over 18 years with an upper arm circumference in the range of 23cm to 33cm (with regard to the supplied arm cuff).

ES

NL

RU

### **MEDICAL LIABILITY NOTE**

PL

These instructions and the described products are not intended as a substitute for the advice provided by your own physician. The information contained herein or the values measured by the product shall not be used for self-diagnosis, self-treatment of a health problem or for self-prescription of medicaments. If you suspect, based on the measurement results, that you have a medical problem, please consult your physician.

TR

AR

## SAFETY INFORMATION

### Special warnings!

- Persons with arrhythmia, diabetes, circulatory disorders or apoplexia must only use this device under medical supervision.
- This product is not suitable for:
  - pregnant women
  - people with arrhythmia
  - people with an intravenous injection
  - during the dialysis treatment
  - women with a preeclampsia (pregnancy poisoning)
- For people who underwent a mastectomy (especially after the removal of lymph nodes), it is recommended to carry out the measurement on the unaffected side.
- In the event of a simultaneous use with other medical electronic devices at the same limb, a pressurisation of the cuff can result in the temporary malfunction of the other devices.
- Results of blood pressure measurements should be interpreted by a physician or a qualified medical professional who is familiar with your case history. Write down your measurement results regularly for your physician and keep your physician up to date regarding the lasting changes of your blood pressure.
- Remember that the blood pressure will certainly vary throughout the day and is influenced by many factors, like stress, eating, smoking, alcohol consumption, medicaments and physical activity, etc. Usually, the blood pressure increases during work and is lowest during sleep.
- Consult your physician regarding specific information on your blood pressure – self-diagnosis and self-treatment using the measurement results may be dangerous. Please follow the instructions of your physician.
- If you suffer from circulatory problems, such as arteriosclerosis, diabetes, liver diseases, kidney diseases, severe hypertension, peripheral circulatory disorders, etc., please consult your physician or a medical professional before using this device.
- Blood pressure measurements which were carried out using this device correspond to those which are obtained by a trained observer using a cuff/stethoscope auscultation method, and are within the accuracy limits which are stipulated in the standard EN 1060-4.
- Modifications to this device are prohibited.
- If this device was modified, a respective inspection and test must be carried out in order to ensure a safe use of the device.
- Only qualified technicians are allowed to repair, dismount and maintain this device. This also applies to possible software upgrades.
- Do not use a mobile phone near the device. This may result in an operational failure.
- Please avoid using the device close to radiating surfaces in order to not falsify the measurements.
- The device is not intended for use in an MR (magnetic resonance) environment.
- Never place the cuff tube around the neck as this may cause asphyxiation.
- Only use approved cuffs, otherwise the correct functionality of the device cannot be guaranteed.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## SAFETY INFORMATION

### Consider before commissioning:

- In order to not falsify the measurement results, please do not carry out measurements in environments where the following environmental conditions exist:
  - Temperatures below 5 °C and over 40 °C
  - Humidity below 15 % and over 93 % relative humidity
  - Air pressure below 700 hPa and over 1060 hPa
- Only for use in indoor spaces.
- When using a device in the presence of children, an attentive supervision is required.
- Never position or use the device in a wet or moist environment.
- Do not use the device under water, e.g. in a shower.
- Never bring the device into direct contact with fire, gas or oxygen as well as hot objects, such as hobs.
- Take every possible precaution to ensure that the device does not fall down or is damaged otherwise.
- If problems at the device occur, absolutely have it repaired immediately.
- Do not lubricate or wash the device.
- Never press the Start/Stop button **(A)** when no cuff is applied.

DE

### **Danger!**

- Do not bring the device into direct contact with water or other liquids.
- The device must not be used outdoors.
- Do not touch the device with wet hands.
- Do not store the device near a sink or a bathtub as it may fall or be pulled into the sink or the bathtub.

EN

FR

### **Warning!**

- Do not leave the device without supervision when there are children or persons not used to handling the device around.
- Make sure that children do not play with the device.
- Only use the device for applications as described in this manual.
- This device is not intended to be used by persons (including children) with limited physical, sensory or mental aptitude or lack of experience and/or knowledge unless they are supervised by a person responsible for their safety or have received instruction from this person as to how the device is used.
- Do not use this device under blankets and pillows as this may lead to excessive heat, fire, injuries or an electrical shock.
- The device must not be used outdoors.
- Do not allow children to play with the packing material as there is a risk of suffocation.
- Never use old and new batteries together.
- Never put heavy objects onto the device.
- Make sure that there are no sharp bends in the tube.

IT

ES

NL

RU

PL

TR

AR

## INTRODUCTION

The **Promed BDP-200** blood pressure meter uses the oscillometric method of blood pressure measurement and is thus particularly suitable for the personal measurement and monitoring of blood pressure. With the oscillometric measurement method the meter measures the flow of your blood through the upper arm artery and converts these into readable measured values. In addition to this, it is not necessary to use a stethoscope with this measurement technique. The **Promed BDP-200** is an automatic blood pressure meter that can measure the systolic, diastolic pressure and pulse. This device is only intended for use by adults. An intelligent control system for the pumping process ensures that the unpleasant feeling caused by incorrect pumping is reduced and shortens the measuring time. In this way, the useful life of the arm cuff is substantially extended. Each measurement result is displayed on the screen and automatically saved. 2x90 memory spaces are available for this. The **Promed BDP-200** possesses a WHO (World Health Organisation) blood pressure classification index with which you can interpret the measured values more easily. Before using the **Promed BDP-200** blood pressure meter, please read through the operating instructions carefully and keep the operating instructions in a safe place.

### What is irregular heartbeat?



The **Promed BDP-200** blood pressure meter can measure blood pressure and pulse even in the event of an irregular heartbeat. An irregular heartbeat is defined as a heartbeat that differs by at least 25% from the average of all the heartbeats during the blood pressure measurement. It is important that you sit still and relaxed during the measurement and do not speak.

**Please note:** If, when measuring, you frequently see the symbol for irregular heartbeat  we recommend that you make sure that you see a doctor.

### What is blood pressure?

Blood pressure is the force exerted by the blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimetres of mercury (mmHg). The natural blood pressure of a person is represented by blood pressure at rest. This is obtained by measuring blood pressure in the morning before getting up.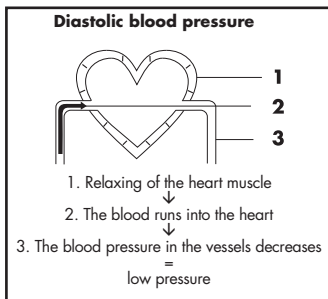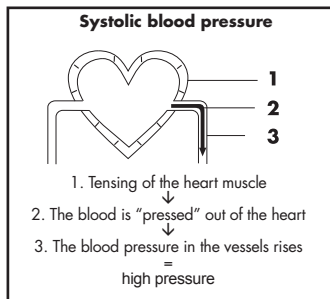

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## INTRODUCTION

### What is high blood pressure and how can this be controlled?

High blood pressure is abnormally high arterial blood pressure which, if it remains uncontrolled, can cause many health problems such as stroke and heart attack. High blood pressure can be regulated by means of lifestyle, the avoidance of stress and medication under medical supervision. In order to prevent high blood pressure or keep it under control, the following principles should be adhered to:

- do not smoke
- regularly do a sport
- eat less salt and fat
- go to the doctor regularly
- keep to the correct weight

### Why should one measure one's blood pressure at home?

Visiting a clinic or doctor's practice can lead to nervousness and thus produce increased blood pressure. The measured values can be 25 to 30 mmHg higher than in a measurement at home. Measuring at home reduces the effects of other influences on the blood pressure measurement and thus supplements the measurements taken at the doctors and show the course of blood pressure more accurately and completely.

### The blood pressure classification index of the WHO

Standards for assessing high blood pressure, without taking age into account, have been specified by the World Health Organisation (WHO) in accordance with the following table.

Blood pressure level		Systol (mmHg)	Diastol (mmHg)	Colour	Recommendations
Grade 3	Sever hypertension	$\geq 180$	$\geq 110$	red	Immediately repeat the measurement. If the result is confirmed, you have a severe hypertension and have to check your blood pressure at least once a day.
Grade 2	Moderate blood pressure	160 ~ 179	100 ~ 109	red	Immediately repeat the measurement. If the result is confirmed, you have a moderate hypertension and have to check your blood pressure several times per week.
Grade 1	Slight blood pressure	140 ~ 159	90 ~ 99	red	Immediately repeat the measurement. If the result is confirmed, you have a slight hypertension and have to check your blood pressure several times per month.
High-normal		130 ~ 139	85 ~ 89	Orange	Your blood pressure is slightly increased. It is recommended to seek advice regarding changes in lifestyle and to check the blood pressure regularly.
Normal		120 ~ 129	80 ~ 84	Yellow	
Optimal		< 120	< 80	Green	

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## OPERATING THE DEVICE

### Checking/replacing the battery

- To replace/insert the batteries, remove the cover of the battery compartment (**J**) and insert 4 batteries 1.5V AAA (**K**). Ensure that the batteries are inserted with their right polarity (+/-). After inserting, close the battery compartment again.
- **Promed BDP-200** checks the charge status of the batteries; the battery status display (10) shows the low capacity of the battery and the  symbol appears. If this is the case, the batteries must be replaced immediately, otherwise the measurement result will be falsified.

#### Note:

When the battery is changed, all stored measured values will be deleted.  
After a battery change, date and time must be set again.

#### Caution:

- Remove the batteries when the device is not used for a longer time.
- Do not use old and new batteries or different battery types at the same time.
- Warning: If batteries leak and come in contact with skin or eyes, immediately clean these areas with plenty of water.
- Batteries shall only be used by adults. Keep the batteries out of the reach of children.
- It is recommended to use batteries of the same or similar type.
- Remove empty batteries from the device.
- Dispose of the batteries according to the instructions of the battery manufacturer.
- Do not use batteries when they have expired.

#### Note:

If the power supply is interrupted or the battery status display (10) lights up during measurement, the measurement must absolutely be repeated.

### Connection of the arm cuff

To connect the arm cuff (**I**) to the device, insert the tube connector (**H**) into the tube connection (**G**). Please ensure that the connector is inserted into the device without using force.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## OPERATING THE DEVICE

### Setting the clock

#### Upon initial commissioning / battery change

- To switch on, press the Start/Stop button **(A)**.
- Now press the Mode button **(C)**. The four-digit year display flashes.
- By pressing the Set button **(B)**, you can set the year.
- By pressing the Mode button **(C)** again, you enter the setting mode for the month. By pressing the Set button **(B)**, you can set the month.
- By pressing the Mode button **(C)** again, you enter the setting mode for the month. By pressing the Set button **(B)**, you can set the day.
- By pressing the Mode button **(C)** again, you enter the setting mode for the hour. By pressing the Set button **(B)**, you can set the hour.
- By pressing the Mode button **(C)** again, you enter the setting mode for the minute. By pressing the Set button **(B)**, you can set the minute.
- By pressing the Mode button **(C)** again, the settings are stored and the device switches to the stand-by mode.

#### Before the measurement

Before you carry out a blood pressure measurement you should remain entirely calm for 5 - 10 minutes, not eat, not drink any alcohol, not smoke, not do any sport and not bathe. All of these factors influence the measurement result. Remove items of clothing that are tight on your upper arm. Always measure on the same arm (normally left). Carry out the measurement regularly at the same time of day, because blood pressure changes during the day.

#### Factors that can lead to incorrect measurements

Everything that supports the arm during measurement can increase the blood pressure. Make sure that you are in a comfortable position and do not move your arm during the measurement. If necessary, you can place a cushion under the arm to be measured. If the arm artery is lower or higher than the heart this will lead to an incorrect measurement result.

#### Note:

Only use manufacturer-approved and clinically approved cuffs. A loose cuff or a defective cuff with an exposed bladder leads to wrong measured values. In the event of repeated measurements, blood is accumulated in the arm arteries which may result in wrong measured values. Continuing blood pressure measurements should be repeated after a waiting period of at least 1 minute. It is important to hold the arm upwards in order to ensure that the accumulated blood can drain.

DE

EN

FR

IT

ES

NL

RU

PL


TR

AR

## OPERATING THE DEVICE

### Putting on the arm cuff

1. Place the arm cuff flat on a table with the hook and loop side facing down. Pass the end of the arm cuff through the metal loop so that a circle is formed. The hook and loop fastener is now visible from the outside.
2. Push the arm cuff over the left upper arm so that the tube lies in the direction of the lower arm.
3. Push the arm cuff over the arm as shown. Make sure that the lower edge of the arm cuff lies about 2 to 3 cm above the elbow and the rubber tube of the arm cuff lies on the inner side of the arm.
4. Pull on the free end of the arm cuff and close the arm cuff using the hook and loop fastener.
5. The arm cuff should now be tight on your upper arm, so that 2 fingers fit between the arm cuff and upper arm. Each item of clothing that is tight on the arm must be removed.
6. Fasten the arm cuff with the hook and loop fastener in such a way that it fits comfortably and not too tightly. Lay your arm on a table (palm upward), so that the arm cuff is at the same height as the heart. Make sure that the tube is not kinked.

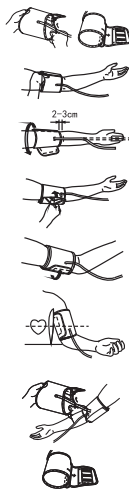### Please note:

If it is not possible to put the arm cuff on the left arm, it can also be placed on the right side. All measurements should, however, be carried out on the same arm.

### Measurement

As soon as the cuff is suitably positioned, you can start the measurement.

Before you can start measuring your blood pressure, you must select the respective use by pressing the Set button **(B)**.

Now press the Start/Stop button **(A)** in order to confirm the selection and to start the measurement.

- For a period of 3 seconds, all symbols are shown on the display. Afterwards, the display shows the value "00". The device is now ready to carry out a measurement and automatically starts inflating the cuff.
- As soon as the device has built up sufficient pressure within the cuff, it automatically stops inflating and starts deflating the air. The actual measuring process starts.
- As soon as the device can measure a pulse beat, this is indicated by a flashing pulse beat display  (8).

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## OPERATING THE DEVICE

- When the measurement is completed, the systolic and the diastolic pressure as well as the measured pulse frequency are displayed and stored. Furthermore, the measurement result, with regard to the WHO scale, is shown on the blood pressure analysis display (4). Now the cuff allows the residual air escape from the system until it is completely empty.
- When the device is not used for more than one minute, it switches off automatically.

### Note:

- When the device determines during inflating that the pressure is not sufficient, it automatically starts a new inflating process.

### Note:

- If the device does not stop inflating, press the Start/Stop button **(A)** which leads to an interruption of the measurement and the immediate reduction of the cuff pressure.

### Note:

- If it is necessary to abort the measurement for whatever reason (e.g. discomfort), this is possible by pressing the Start/Stop button **(A)** at any time. This interrupts the measurement and immediately reduces the cuff pressure.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

### Using the memory function

This device has a memory for the storage of 40 measurements for each user. Each time a measurement is completed, the device automatically stores the blood pressure as well as the pulse frequency. If 40 data sets have already been stored, the oldest measured values are replaced by the new ones.

### Reading the measured values

- Select the respective user by pressing the Set button **(B)**.
- By pressing the Memory button **(D)**, you enter the memory menu.
- If no measured values are stored, the display only shows date and time. Otherwise, the display shows the average of the last 3 measurements and the symbol for average values (12).
- By pressing the Memory button **(D)** again, you can read the individual measured values (the last value is displayed first, the first value is displayed last).
- To exit the memory menu, press the Start/Stop button **(A)** and the device switches to the stand-by mode.

### Deleting measured values

- Select the respective user by pressing the Set button **(B)**.
- By pressing the Memory button **(D)**, you enter the memory menu.
- By pressing and holding the Mode button **(C)** and the Set button **(B)** at the same time, you delete all measured values of the selected user.

### Note:

Deleting of measured values cannot be reversed. Power tools, accessories and packaging are to be recycled in an environmentally friendly manner.

## TROUBLESHOOTING

Error/ Error Display	Cause	Corrective action
The display is empty, even though the Start/ Stop button (A) is pressed.	The batteries are empty.	Replace the batteries by new ones.
	The batteries are not inserted correctly / the polarity (+/-) is wrong.	Insert the batteries correctly.
 This error code is displayed when the measured blood pressure value is excessively low or high.	The cuff is not attached correctly.	Attach the cuff correctly.
	You talked during measurement.	Repeat the measurement and avoid moving and talking.
	You moved yourself or your arm during measurement.	
 This error code is displayed when there is a fault in the air circulation of the device.	The cuff is not connected correctly with the device.	Check all connections between the cuff and the device and repeat the measurement.
 This error code is displayed when too much pressure is applied to the cuff.	The pressure in the cuff exceeds 300mmHg.	Switch off the device and repeat the measurement.
 This error code is displayed when there is a fault during measurement.	No measured valued can be determined.	Switch off the device, attach the cuff correctly and repeat the measurement.

When the error code "EP" is displayed, please contact the manufacturer.

If the fault persists, please contact the manufacturer before trying to repair the device on your own.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## CARE / MAINTENANCE / STORAGE

- Clean the device with a soft dry cloth.
- Do not use abrasives or volatile cleaning agents.
- Do not immerse the device or parts thereof in water.
- Thoroughly wipe off all surfaces of the blood pressure monitor and the cuff and ensure that you clean the inner and outer surface of the cuff. Ensure that no moisture can enter the main device.
- For cleaning, you can use a dilution of neutral cleaning agents or soap solution.
- Carefully wipe off with a dry cloth in order to remove excess moisture which may remain on the blood pressure cuff. Lay down the cuff unrolled in your apartment in order to allow it to air-dry.
- Ensure that the cuff is completely dry before using it again.

### Note:

- For cleaning, use only tap water.
- Do not bring the device into direct contact with water or other liquids.
- Switch off the device when it is not in use.
- Before storing the **Promed BDP-200** for a longer period, remove the batteries from the battery compartment. Leaking batteries may damage the device.
- Always store and transport the device after use in the storage bag.
- Store the device and its accessories in a cool, dry place.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- For inspection and recalibration, the device does not have to be sent to the dealer or the manufacturer.
- If you detect other problems, please contact your dealer who sends in the device, if necessary. Never try to correct an error on your own.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## DISPOSAL

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

### Only for EU countries:

Do not throw power tools in with the domestic waste! In accordance with the European Directive 2012/19/EU on waste electrical and electronic equipment and its implementation into national law, nonfunctioning power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable material that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices using the appropriate collection systems or send the device to the place of purchase for disposal. The place of purchase will then recycle the device.

### Rechargeable batteries/batteries:

Do not throw your rechargeable batteries/batteries in the domestic waste, in fire, or in water. Rechargeable batteries/batteries should be collected, recycled, or disposed of in an environmentally friendly manner.

## DISPOSAL

### Only for EU countries:

According to the directive 91/157/EEC, defect or used rechargeable batteries/batteries must be recycled. Waste rechargeable batteries/batteries can be directly handed to:

Promed GmbH, Lindenweg 11, D-82490 Farchant.

## Warranty conditions

This product was produced with the utmost care and was thoroughly tested before it left the factory. Therefore, we provide a warranty period of 36 months from the date of purchase for this product. **Promed** products conform to their description and the relevant specifications; it is your responsibility to ensure that the product which you have purchased is designed for your intended use.

- In the event of material defects and/or manufacturing errors which appear during proper use and are detected during the warranty period, we will replace all defective parts of the product free of charge within the warranty period, including the labour costs for warranty repairs.
- **The warranty does not cover:**
  - Normal wear on the product
  - Defects which were caused during transportation or storage of the product
  - Defects or damage which were caused through improper use or inadequate maintenance
  - Damage resulting from disregarding the notes in the operating manual
  - Damage resulting from modifications to the product which were not carried out by **Promed**.
  - Damage caused by sharp objects or as a result of torsion, compression, being dropped, an abnormal impact or other actions which are beyond **Promed's** reasonable control.
  - Parts which are subject to wear (e.g. moving parts such as ball bearings, etc., fasteners, etc.) are generally excluded from the warranty.
- **A claim under the warranty is not enforceable if:**
  - the product is not sent back in its original packaging or in adequately secure packaging;
  - it has been modified or repaired by a person or company other than **Promed** or one of **Promed's** authorised dealers;
  - the product has been repaired using parts which are not approved by **Promed**;
  - the serial number / LOT number of the product has been removed, deleted, changed or made illegible.
- For reasons of hygiene, any products which have been exposed to direct contact with a body or with bodily fluids (e.g. blood) must be packaged in an additional plastic bag before they are sent back. In this case, a specific note of this situation must be made in the package or in the accompanying documents.
- For components or products which are replaced during the warranty period, a warranty is only provided for the remainder of the original warranty period; provided that the product or part replacement was carried out by **Promed** or one of **Promed's** authorised dealers.
- The warranty period begins on the day of purchase. Claims under the warranty must be established within the warranty period. Complaints arising after the expiry of the warranty period cannot be considered.

DE

EN

FR

IT

ES

NL

RU

PL

TR

AR

## WARRANTY CONDITIONS

- Under these warranty conditions, the warranty shall only apply if the date of purchase can be proven by means of a sales receipt or similar.
- We reserve the right to make technical and visual changes as well as changes to the features.
- This warranty is only valid and enforceable in the country in which it was purchased by the original buyer, provided that it was **Promed**'s intention that the product should be offered for sale in this country. This warranty is also enforceable in any country within the European Economic Area in which **Promed** has an authorised importer or sales partner. Depending on each country, specific and differing warranties and guarantees may be applicable owing to the particular applicable legislation. These legal provisions are neither excluded nor limited by these warranty conditions. To the extent permissible under national law, the warranty period shall not be extended, renewed or otherwise affected by any subsequent resale, repair or replacement of the product.
- The provisions of the UN Convention on Contracts for the International Sale of Goods shall not apply.
- Our warranty conditions do not affect the seller's legal warranty obligation.
- To the fullest extent possible under the applicable mandatory legislation, these warranty conditions constitute your sole and exclusive legal remedy and shall apply in place of any other express or implied warranty conditions. **Promed** is not liable for any unusual, incidental, punitive or consequential damages including, but not limited to loss of profits, loss of use, loss of revenue, costs of replacement equipment or facilities, third party insurance claims, and damage to property which are the result of the purchase or use of the product or occur owing to a breach of warranty, a breach of contract, negligence, product defects or any other legal or statutory circumstances, even though **Promed** was aware of the possibility of such damages. **Promed** is not liable for any delay in warranty claims.
- **Promed** cannot be held liable for any possible translation errors.

The following information is essential for smooth processing:

1. Original proof of purchase/receipt or dealer's stamp with date of purchase
2. Defect discovered
3. Product designation / type / serial/batch number

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**EN |**

The requirements of the EU Directive 93/42/EEC in conjunction with the EU Directive 2007/47/EC for class IIa medical devices were met according to the standard EN 60601-1-2:2007.

CE according to the EU Directives 2014/35/EU, 2014/30/EU, 2011/65/EU, 1907/2006/EC, 2005/69/EC, 93/68/EEC and 2001/95/EC.

The CE marking for the device refers to the EU Directive 93/42/EEC in conjunction with the EU Directive 2007/47/EC. Device classification: Class IIa.

Furthermore, the device fulfils the requirements of the standards:

- IEC 60601-1:2012
- EN 60601-1:2006 +A1:2013
- IEC 60601-11-11:2010
- EN 60601-1-11:2010
- EN 1060-1:1995
- +A2:2009
- EN 1060-3:1997
- +A2:2009
- EN 1060-4:2004

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Transportation only in the original  
packaging

**Transportation/storage  
conditions**

Transportation and storage  
temperature:  
-25°C – +70°C  
Relative humidity:  
<= 93% non-condensing  
Air pressure:  
700 to 1060hPa

**Operating conditions:**

Operating temperature:  
+5°C – +40°C  
Relative humidity:  
15% to 93% non-condensing  
Air pressure:  
700 to 1060hPa

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EN

A sudden change in temperature  
is to be avoided due to the  
possible condensation of the  
device. Use the device only after  
a temperature equalisation.

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