promed HGP-30

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promed HGP-30
 EN | INSTRUCTION LEAFLET
 WRIST BLOOD PRESSURE MONITOR


BEFORE COMMISSIONING

About this device

- A.** Start / stop button
B. Memory button
C. Date / time button
D. User button
E. LCD display
F. WHO scale
G. Battery compartment
H. Batteries
I. Storage box
J. LCD display

LCD display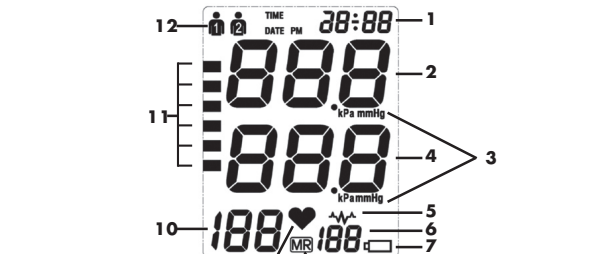

- 1.** Date display
2. Systolic blood pressure
3. Selected measurement unit
4. Diastolic blood pressure
5. Display for irregular heartbeat
6. Storage space display
7. Battery status display
8. Memory symbol
9. Display for pulse
10. Pulse rate display
11. Blood pressure analysis display
12. User display

SCOPE OF DELIVERY / ACCESSORIES

The following components are included in the scope of delivery of your **Promed HGP-30** blood pressure monitor:
 - 1x blood pressure monitor with wrist cuff (circumference 135 – 155 mm)
 - 2x batteries, 1.5 VDC AAA
 - 1x storage box
 - 1x Instructions for use

You can buy the following spare parts at Promed:
 - batteries, 4x 1.5 VDC AAA, part no.: 359994
 - storage adapter, part no.: 964004
 - blood pressure pass, part no.: 984006

DESCRIPTION OF THE DEVICE

Please keep the instructions carefully!

Please read the user manual thoroughly before using the device for the first time. Many thanks for purchasing the **Promed HGP-30** blood pressure meter. You have acquired a high-quality medical product for the personal checking of health.
Promed is a leading company with decades of experience in personal hygiene, wellness and health areas.
 The **Promed HGP-30** unit has been designed and manufactured in accordance with the Medical Device Directives 93/42/EEC to guarantee quality during use. The device may be used once this user manual has been read.
 As the manufacturer, cannot be made liable in any way for injury or damages to people or objects that arise from failure to comply with this user manual. We wish you a lot of enjoyment with your new **Promed HGP-30**. In the following we would like to familiarise you with the **Promed HGP-30** blood pressure meter. Please read the user manual thoroughly before using the device for the first time. You can receive professional advice wherever **Promed** products are sold or get in touch with us if you have any questions. We can provide you with the name of the representative responsible for you.

Promed GmbH Kosmetische Erzeugnisse
 Lindenweg 11, D-82490 Farchant, Ph: +49 [0] 8821/9621-0,
 Fax: +49 [0] 8821/9621-21, E-Mail: info@promed.de

Further informations can be found on our homepage: www.promed.de

PROPER USE

The **Promed HGP-30** is a blood pressure monitor with an integrated time / date display which automatically measures systolic and diastolic blood pressure using the oscillometric measuring method. In addition, the pulse is also measured and, if applicable, an irregular heartbeat is shown. The device is only intended for use on humans. The **Promed HGP-30** has been developed for use at home and should only be used by adults over 18 years of age with a wrist circumference of 125 mm – 215 mm.

MEDICAL LIABILITY NOTE

These instructions and the described products are not intended as a substitute for the advice provided by your own physician. The information contained herein or the values measured by the product shall not be used for self-diagnosis, self-treatment of a health problem or for self-prescription of medications. If you suspect, based on the measurement results, that you have a medical problem, please consult your physician.

SAFETY INFORMATION

Special warnings!

- This device is only intended for use by adults.
- This device is intended for the non-invasive measurement and monitoring of arterial blood pressure. It is not intended for use on extremities other than the wrist or for functions other than blood pressure measurement.
- If you are taking any medication, consult your physician to determine the time of day that is best suitable for measuring your blood pressure. Never change any prescribed medication without consulting your physician.
- Read these instructions for use carefully before using the product in order to avoid measurement errors.
- The device is not an AP / AFG device and is not suitable for use in presence of a flammable anaesthetic mixture with air, oxygen or laughing gas.
- The operator must not touch the poles of batteries and the patient at the same time.
- The user must ensure the safe function and the flawless state of the device before use.
- Only use accessories that are specified / approved by the manufacturer. Otherwise, this may result in damage to the device or danger for the user / patient.
- This device is not suitable for continuous monitoring during medical emergencies or surgeries. Otherwise, the patient's wrist and fingers will become narcotic, swollen and even purple due to a lack of blood.
- The device does not need to be calibrated together with the service every two years.
- When the device is used for measurements on patients with frequent arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, this may result in deviations of the result. Please consult your physician on the result.
- Too frequent or continuous measurements may lead to disturbed circulation and injuries.
- Do not use the cuff on a wound, since this may lead to further injury.
- Do not inflate the cuff on a limb that is monitored by other medical devices at the same time since this may result in a temporary loss of function of the devices used at the same time as well as in an impairment of the patient's blood circulation.

- Persons with arrhythmia, diabetes, circulatory disorders or apoplexia must only use this device under medical supervision.
- This product is not suitable for:
 - pregnant women
 - people with arrhythmia
 - people with an intravenous injection
 - during the dialysis treatment
 - women with a pre-eclampsia (pregnancy poisoning)
- For people who underwent a mastectomy (especially after the removal of lymph nodes), it is recommended to carry out the measurement on the unaffected side.
- The blood pressure level can fluctuate, even in the case of healthy individuals. It is important that comparable measurements are always taken in the same conditions (in a resting state)! If, in spite of considering all these factors, the fluctuations are greater than 1.5 mmHg, and / or the arrhythmia symbol (S) is displayed, please consult your physician.
- In the event of a simultaneous use with other medical electronic devices at the same lib, a pressurisation of the cuff can result in the temporary malfunction of the other devices.
- Results of blood pressure measurements should be interpreted by a physician or a qualified medical professional who is familiar with your case history. Write down your measurement results regularly for your physician and keep your physician up to date regarding the lasting changes of your blood pressure.
- Remember that the blood pressure will certainly vary throughout the day and is influenced by many factors, like stress, eating, smoking, alcohol consumption, medicaments and physical activity, etc. Usually, the blood pressure increases during work and is lowest during sleep.
- Consult your physician regarding specific information on your blood pressure – self-diagnosis and self-treatment using the measurement results may be dangerous. Please follow the instructions of your physician.
- If you suffer from circulatory problems, such as arteriosclerosis, diabetes, liver diseases, kidney diseases, severe hypertension, peripheral circulatory disorders, etc., please consult your physician or a medical professional before using this device.
- Blood pressure measurements which were carried out using this device correspond to those which are obtained by a trained observer using a cuff/stethoscope auscultation method, and are within the accuracy limits which are stipulated in the standard EN 1060-4.
- Modifications to this device are prohibited.
- If this device was modified, a respective inspection and test must be carried out in order to ensure a safe use of the device.
- Only qualified technicians are allowed to repair, dismount and maintain this device. This also applies to possible software upgrades.
- Please keep the device out of proximity to radiating surfaces (e.g. mobile phones, microwave ovens) so that the measurements are not disturbed.
- The device is not intended for use in an MR (magnetic resonance) environment.
- Only use approved cuffs, otherwise the correct functionality of the device cannot be guaranteed.

Consider before commissioning:

- Only for use in indoor spaces.
- When using a device in the presence of children, an attentive supervision is required.
- Never position or use the device in a wet or moist environment.
- Do not use the device under water, e.g. in a shower.
- Never bring the device into direct contact with fire, gas or oxygen as well as hot objects, such as hobs.
- Take every possible precaution to ensure that the device does not fall down or is damaged otherwise.
- If problems at the device occur, absolutely have it repaired immediately.
- Do not lubricate or wash the device.
- Never press the Start/Stop button (A) when no cuff is applied.
- This device contains sensitive components and must be handled with care. Please observe the storage and operating requirements described in the chapter "TRANSPORT AND STORAGE CONDITIONS"!

- Danger!**
 - Do not bring the device into direct contact with water or other liquids.
 - The device must not be used outdoors.
 - Do not touch the device with wet hands.

- Warning!**
 - Do not leave the device without supervision when there are children or persons not used to handling the device.
 - Make sure that children do not play with the device.
 - Only use the device for applications as described in this manual.
 - This device is not intended to be used by persons (including children) with limited physical, sensory or mental aptitude or lack of experience and/or knowledge unless they are supervised by a person responsible for their safety or have received instruction from this person as to how the device is used.
 - The device must not be used outdoors.
 - Do not allow children to play with the packing material as there is a risk of suffocation.
 - Never use old and new batteries together.
 - Never put heavy objects onto the device.

INTRODUCTION


The **Promed HGP-30** blood pressure monitor uses the oscillometric method of the blood pressure measurement and is thus particularly suitable for the personal blood pressure measuring and monitoring. With this measuring method, the device measures the flow of your blood through the arm artery and converts this into readable measured values. Furthermore, this measuring technologies does not require the use of a stethoscope. **Promed HGP-30** is an automatic blood pressure monitor which is able to measure the systolic and the diastolic pressure as well as the pulse. This device is only intended for the use by an adult. An intelligent control of the inflation ensures that the uncomfortable feeling in the event of an incorrect inflation is reduced and that the measurement time is decreased. Thus, the lifetime of the wrist cuff is significantly increased. Each measurement result is shown on the display and stored automatically. 2x120 memory locations are available for this. **Promed HGP-30** has a WHO (World Health Organization) blood pressure classification index which enables you to interpret the measured values more easily.

What is irregular heartbeat?
 The **Promed HGP-30** blood pressure meter can measure blood pressure and pulse even in the event of an irregular heartbeat. An irregular heartbeat is defined as a heartbeat that differs by at least 25% from the average of all the heartbeats during the blood pressure measurement. It is important that you sit still and relaxed during the measurement and do not speak.

Please note! If, when measuring, you frequently see the symbol for irregular heartbeat  we recommend that you make sure that you see a doctor.

What is blood pressure?

Blood pressure is the force exerted by the blood against the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands. Blood pressure is measured in millimetres of mercury (mmHg). The natural blood pressure of a person is represented by blood pressure at rest. This is obtained by measuring blood pressure in the morning before getting up.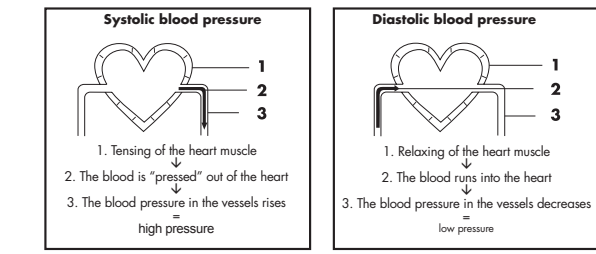

What is high blood pressure and how can this be controlled?

High blood pressure is abnormally high arterial blood pressure which, if it remains uncontrolled, can cause many health problems such as stroke and heart attack. High blood pressure can be regulated by means of lifestyle, the avoidance of stress and medication under medical supervision. In order to prevent a high blood pressure or keep it under control, the following principles should be adhered to:

- do not smoke
- regularly do a sport
- eat less salt and fat
- go to the doctor regularly
- keep to the correct weight

Why should one measure one's blood pressure at home?

Visiting a clinic or doctor's practice can lead to nervousness and thus produce increased blood pressure. The measured values can be 25 to 30 mmHg higher than in a measurement at home. Measuring at home reduces the effects of other influences on the blood pressure measurement and thus supplements the measurements taken at the doctors and show the course of blood pressure more accurately and completely.

What are normal values?

In a resting state, the blood pressure is too high when the diastolic pressure is over 90 mmHg and / or the systolic blood pressure is over 160 mmHg. If this is the case, please consult your physician immediately. Long-term values at this level will endanger your health because of the accompanying progressive damage to the blood vessels in your body. If the systolic blood pressure value is between 140 mmHg and 160 mmHg and / or the diastolic blood pressure value is between 90 mmHg and 100 mmHg, please also consult your physician. Regular self-checks are also necessary. If your blood pressure values are too low, meaning systolic values under 100 mmHg and / or diastolic values under 60 mmHg, please also consult your physician. Even if your blood pressure values are normal, it is also recommended to conduct regular self-checks with your blood pressure monitor. This allows you to recognise any changes to your values at an early stage and react accordingly. If you are undergoing medical treatment to control your blood pressure, make a note of your blood pressure level by taking self-measurements at determined times of day and show these values to your physician. Never use the results of your measurements on their own to change the dose of medication prescribed by your physician.

The blood pressure classification index of the WHO

Standards for assessing high blood pressure without taking into account the age of the individual were determined by the World Health Organisation (WHO) in accordance with the following table.

Blood pressure level	Systolic (mmHg)	Diastolic (mmHg)	Recommendations
Hypotension	< 100	< 60	Consult your physician
Ideal	100 - 120	60 - 80	
Normal	120 - 130	80 - 85	
Pre-high blood pressure	130 - 140	85 - 90	Consult your physician
High blood pressure	140 - 160	90 - 100	Seek medical advice
Very high blood pressure	160 - 180	100 - 110	Seek medical advice
Dangerously high blood pressure	> 180	> 110	Seek medical advice immediately!

- Note:**
 - If your values are usually in the ideal or normal level when resting, but unusually high during physical or mental stress, you may suffer from so-called "labile hypertension". If you think that this is the case, please consult your physician.
 - Diastolic blood pressure of over 110 mmHg when measured correctly requires immediate medical care.

OPERATING THE DEVICE

Replacing / inserting the battery

- To replace and / or insert the batteries, remove the cover of the battery compartment (H) and insert 2 1.5 V AAA batteries (I). While doing so, make sure that the batteries are inserted in the correct polarity (+/-). After inserting, close the battery compartment again.
- The **Promed HGP-30** checks the charge status of the batteries. If these are too weak, this is displayed via the battery status (7) sign and the symbol  appears. If this appears, the batteries must be replaced immediately, otherwise the measurement will not be correct.

- Note:**
 - After the battery warning appears, the device is blocked until the batteries are replaced.

- Note:**
 - After changing the batteries, the date and time must be set again.

- Caution:**
 - Remove the batteries if the device is not going to be used for a long period of time.
 - Do not mix old and new batteries or different battery types.
 - Warnings!** If batteries leak and come into contact with skin or eyes, wash these areas immediately with plenty of water.
 - Batteries are not toys. Store batteries out of reach of children.
 - Only batteries of the same or similar type are recommended.
 - Remove empty batteries from the device.
 - Dispose of the batteries in accordance with the battery manufacturer's instructions.
 - Do not use batteries after their expiry date.

- Note:**
 - If the power supply to the device is interrupted during a measurement or the battery status display (7) lights up, the measurement must be repeated.

Setting the user, date, time and measurement unit
User selection

With the **Promed HGP-30** blood pressure monitor, 2 individuals can measure and store their blood pressure values independently from each other.

- Before taking the measurement, ensure that you have selected the desired user.
- To display the set user, press the date / time button (C) and hold it down for at least 3 seconds. The set user then flashes on the display. To confirm, press the start / stop button (A).
- To select the user, press the memory button (B).
- It is recommended that you save the first person to use the device as user 1.
- To change user, press the user button (D). The user display (12) then flashes. By pressing the user button (D) again, you can change the user.

Reading the set data

To read the set values, press the date / time button (C) once.

Setting the time and date

The **Promed HGP-30** has an integrated clock with date display. This has the advantage of allowing not just the blood pressure values to be stored for each measurement, but also the exact time the measurement was taken.

After new batteries have been inserted, the clock begins with the following setting: 2010-06-20 09:30. You must then enter the date and current time again.

- Press the date / time button (C) for at least 3 seconds until the user display (12) flashes. Now, press the date / time button (C) again. The set year then flashes on the display.

- The correct year can then be entered by pressing the memory button (B).

- Now, press the date / time button (C) again. The set day then flashes on the display.

- The correct month can then be entered by pressing the memory button (B).

- Now, press the date / time button (C) again. The set day then flashes on the display.

- The correct day can then be entered by pressing the memory button (B).

- Now, press the date / time button (C) again. The set time (hours) then flashes on the display.

- The correct time (hours) can then be entered by pressing the memory button (B).

- Now, press the date / time button (C) again. The set time (minutes) then flashes on the display.

- The correct time (minutes) can then be entered by pressing the memory button (B).

Setting the measurement unit

- Now, press the date / time button (C) again. The set measurement unit (mmHg) then flashes on the display.

- The measurement unit can then be changed to kPa by pressing the memory button (B).

- After you have made all changes, press the date / time button (C) again. Now, the date and then the time are briefly displayed. The entry is now saved and the clock will start.

- Note:**
 - Each time you press the buttons (date / time button (C), memory button (B)), there is an entry (e.g. switching from hour to minute mode or changing a value by +1). However, if you keep the corresponding button pressed, you can change this quicker, or set the desired value.

Before the measurement

Comparable blood pressure measurements must always be taken under the same conditions! These are usually taken during periods of rest. Before taking a blood pressure measurement, you should be absolutely still for 10 minutes, do not eat, do not drink alcohol, do not smoke, do not exercise and do not bathe. All these factors influence the measurement result. Remove all items of clothing that are tight-fitting on your arm. Always use the same arm (usually the left arm). Take the measurement at the same time of day, as your blood pressure changes during the day.

Factors that can lead to inaccurate measurements
 All efforts made by the patient to support the arm can increase blood pressure. Ensure that you are in a comfortable, relaxed position and do not activate any muscles in the measurement area during the measurement. If required, use a cushion for support.

- The performance of the **Promed HGP-30** blood pressure monitor can be influenced by extreme temperatures, moisture and altitude.
- If the artery in the wrist is significantly lower (higher) than the heart, a higher (lower) inaccurate blood pressure will be measured! [Each 15 cm difference leads to a measurement error of 10 mmHg!]
- If repeated measurements are carried out, blood can gather in the respective wrist, which can lead to inaccurate results. For this reason, correct blood pressure measurements should be repeated after waiting 5 minutes or after holding the arm up for at least 3 minutes so that the gathered blood can disperse.
- A loose cuff or a defective cuff with an exposed inflation bladder leads to inaccurate measurement results.

- Note:** Use the **Promed HGP-30** only if the cuff is intact.

- Note:** Do not use the **Promed HGP-30** on a wrist where wounds or other injuries are present.

- Note:** Only use the **Promed HGP-30** on the wrist.

Putting the cuff on

- Remove all items (wristwatch, bracelet, etc.) from your wrist. If your physician diagnosed poor circulation of one wrist, please use the other one.
- Remove any items of clothing from your wrist to make sure that the device is positioned directly on your skin.
- Put the cuff on the wrist with the hook-and-loop side facing downwards so that the display is on the inner side of the arm. No clothes may be between the cuff and the wrist. Any item of clothing that is tight-fitting on the arm must be removed.
- Make sure that the lower edge of the cuff is positioned at a distance of about 1 - 1.5 cm from the lower edge of your palm.
- Secure the cuff using the hook-and-loop fastener in such a way that it fits comfortably and is not too tight.
- Sit comfortably with your legs not crossed, your feet flat on the ground, your back and wrist supported and place your arm on a table (palm facing upwards) so that the cuff is at the level of your heart. This is particularly important for hypertensive patients.

- Note:**
 - If it is not possible to put the cuff on the left arm, it can also be placed on the right side. However, all measurements should be carried out on the same arm.

Measurement

After the cuff has been suitably positioned, the measurement can begin.

- By pressing the start / stop button (A), you can start the measurement process and the cuff begins to inflate. For this, the increasing cuff pressure is shown on the display.

- After maximum pressure is reached, the pump process ends and the pressure gradually decreases. The cuff pressure is displayed during the measurement. Once the device measures the pulse, the heart symbol on the display starts to flash for each heartbeat.

- Once the measurement is complete, the measured systolic and diastolic pressure as well as the pulse frequency is displayed. Example (fig.): systolic 118, diastolic 73, pulse 75.

The measurement results are displayed until you switch off the device. When no button is pressed for 3 minutes, the device switches itself off automatically to save the batteries.

Aborting the measurement

If it is necessary to abort a blood pressure measurement for any reason (e. g. discomfort), this is possible by pressing the start / stop button (A) at any time.

Using the memory function


This device has a memory for storing 120 measurements for each user. Every time a measurement is completed, the device automatically saves the blood pressure and the pulse frequency.

- Reading the measured values**
 - Select the user you desired by pressing the user button (D).

- By pressing the memory button (B) you will reach the storage menu.

- If no measurement data has been saved, the display does not show anything apart from the date and time. Otherwise, the display shows the average of the last 3 measurements.

- By pressing the memory button (B) once more, you can read the individual measured values (the latest measured value is displayed first and the first displayed last). The respective storage space number is displayed by the memory symbol (B) (last measured value "MR 1", all previous measured values "MR 2" – "MR 120").

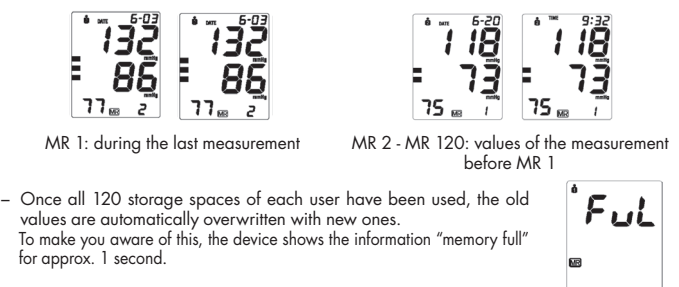- Once all 120 storage spaces of each user have been used, the old values are automatically overwritten by the new ones. To make you aware of this, the device shows the information "memory full" for approx. 1 second.

Deleting measured values

Before you delete all the measured values, ensure that you will not need to use these measured values at a later point in time. It is wise to record measured values in writing (e.g. in the accompanying blood pressure record card), and this can give your physician additional information at your visit.

To delete all stored measured values, hold down the memory button (B) for at least 5 seconds. The display now shows the symbol "CL". When you let go of the button, the "CL" display starts to flash and, when you press the memory button (B) again, all stored measured values are deleted.

- Note:**
 - The deletion of measured values cannot be reversed.

TROUBLESHOOTING

If an error occurs during a measurement, the measurement is interrupted and a corresponding error code is shown (example: error no. 2).

Error no.	Possible cause(s)
ERR 1	No pulse is recognised.
ERR 2	Unnatural pressure impulses have influenced the measurement result. It is possible that the arm / hand was moved during the measurement.
ERR 3	The inflation process is taking too long. The cuff may not be positioned correctly.
ERR 5	The measured values showed an intolerable difference between the systolic and diastolic pressure. Read the instructions again carefully. Contact your physician if you continue to obtain unusual results.

Other possible faults and their remedies

If problems arise when using the device, check the following points and take the corresponding measures if necessary:

Fault	Remedy
The display remains clear when the device is switched on, although the batteries have been inserted.	1. Check the polarity (+/-) of the batteries is correct and fix, if applicable. 2. If the display looks unusual, replace or exchange the batteries.
The device often does not measure the blood pressure values or the measured values are too low (too high).	1. Check the cuff position. 2. Measure the blood pressure again while taking into account the information given above.
Each measurement shows a different value, although the device is working normally and the displayed values are normal.	1. Please check all points mentioned in the chapter "TROUBLESHOOTING". 2. Repeat the measurement. Please be aware: the blood pressure is always fluctuating, which means that there is a certain variability in consecutive measurements.
The measured blood pressure is different from values measured by the physician.	Take note of the daily development of your values and consult your physician. Please be aware: individuals visiting their physician frequently experience a state of anxiety which can lead to a higher reading value at the physician than being treated at home in more comfortable conditions.

- If you cannot remedy the fault, contact the manufacturer before attempting to repair the device yourself.

CARE / MAINTENANCE / STORAGE

- Clean the device with a soft dry cloth.
- Do not use abrasives or volatile cleaning agents.
- Do not immerse the device or parts thereof in water.
- Thoroughly wipe off all surfaces of the blood pressure monitor and the cuff and ensure that you clean the inner and outer surface of the cuff. Ensure that no moisture can enter the main device.
- For cleaning, you can use a dilution of neutral cleaning agents or soap solution.
- Carefully wipe off with a dry cloth in order to remove excess moisture which may remain on the blood pressure cuff. Lay down the cuff unrolled in your apartment in order to allow it to air-dry.
- Ensure that the cuff is completely dry before using it again.

- Note:**
 - For cleaning, use only tap water.
 - Do not bring the device into direct contact with water or other liquids.
 - Switch off the device when it is not in use.
 - Before storing the **Promed HGP-30** for a longer period, remove the batteries from the battery compartment. Leaking batteries may damage the device.
 - Always store and transport the device after use in the storage bag.
 - Store the device and its accessories in a cool, dry place.
 - Do not expose the device to direct sunlight to protect it against dirt and moisture.
 - For inspection and recalibration, the device does not have to be sent to the dealer or the manufacturer.
 - If you detect other problems, please contact your dealer who sends in the device, if necessary. Never try to correct an error on your own.

DISPOSAL

Power tools, accessories, and packaging should be recycled in an environmentally suitable manner.

Only for EU countries:

Do not throw power tools in with the domestic waste! In accordance with the European Directive 2012/19/EU on waste electrical and electronic equipment and its implementation into national law, nonfunctioning power tools must be collected separately and recycled in an environmentally suitable manner.

This symbol indicates that disposal of this product in the domestic waste is not permitted within the EU. Waste devices contain valuable recyclable materials that should be recycled. Recycling also prevents uncontrolled waste disposal from damaging the environment and human health. Therefore, please dispose of all waste devices using the appropriate collection systems or send the device to the place of purchase for disposal. The place of purchase will then recycle the device.

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