English

Thank you for purchasing EAR ZOOM[®] PRO sound booster. For best results, read all the instructions carefully before using the device for the first time. Ear Zoom[®] PRO runs on two AAA batteries (not included).

Safety Warnings

Keep device out of reach of children.

Be careful when placing the device on your ear. Do so gently so you do not hurt your ear.

Warning: Batteries (battery pack or batteries installed) shall not be exposed to excessive heat such as sunshine, fire or the like.

IMPORTANT! Always start with the volume low and increase it gradually to avoid hearing loud, unpleasant noise that may damage your hearing.

Always make sure to lower the volume when a sound or a person is getting closer to you. Over short distances the sound will be amplified and could be too loud.

If the unit is not amplifying the sound well, the batteries may be very low.

Ear Zoom[®] Pro is not a medical device or a device to correct hearing.

IMPORTANT! This product is not water resistant. Keep it away from water or the device could be damaged. If the unit comes into contact with water, remove the batteries and put the device in a warm place to dry.

Usage temperature range: Do not expose the product to temperatures below 0°C or above 50°C.

Attention should be drawn to the environmental aspects of battery disposal.

Product Description:

- A. Earphones
- B. ON/OFF button
- C. Volume up button
- D. Volume down button
- E. Earphone jack
- F. Clip
- G. Battery compartment
- H. Display screen
- I. Microphone

Battery installation:

To insert the batteries, press down on the battery compartment cover (G), as shown in Figure 2.

Once you have removed the cover, insert the AAA batteries following the polarity markings indicated inside the battery compartment. Close the battery compartment (G), as shown in Figure 4.

Instructions for Use:

To ensure optimum performance of the product, make sure the microphone is always exposed and not covered.

1. To switch on your Ear Zoom[®] Pro, press the ON/OFF button (B).

2. The screen (H) will turn on and display the chosen volume level. There are 16 volume levels. Level 16 is the maximum volume level.

Important:

Always start with the volume low and increase gradually to avoid hearing a loud, unpleasant noise that may damage your hearing.

3. Set the volume to level 1 by pressing the volume down button (D).

4. Plug the earphones (A) into the earphone jack (E). Ensure that they are firmly plugged in.

5. Place the earphones in your ears. 6. Adjust the volume to the desired level by pressing the volume up button (C) and the volume down button (D). Remember to turn down the volume when you are near other people.

7. To switch off your Ear Zoom[®] Pro, press the ON/OFF button (B).

Quality Warranty:

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country. This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

CAUTION: DISPOSAL OF MATERIALS

A symbol of a crossed-out wheeled bin means you should find out about and follow local regulations about disposing of this kind of product. Do not dispose of this product as you would other household waste. Dispose of this device in accordance with the corresponding local regulations. Electrical and electronic devices contain hazardous substances that can have harmful effects on the environment and/or human health and should be recycled properly.

Australia & New Zealand only

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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