Imported by - Importé par:
GD import (Genius Ideas & Welllys)
190 chemin de la Frayère - 06530 Peymeinade - France
www.gdimport.com
www.genius-ideas.com
www.wellys-care.com

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GB – NASAL & FLOSSER WATER JET “PUSH & CLEAN” - 209027

No batteries or cord are needed. Easy to carry when travelling, super lightweight and portable. Supplied with two different nozzles (with silicone head for nasal care, without silicone head for oral care).

Nasal Cavity care – FIG. A

Warm salt water washes nasal cavity, clean viruses and bacteria. Frequent nasal rinse significantly reduces nasal allergies, stuffiness, dryness etc.

Oral Care – FIG. B

Remove food debris and harmful bacteria, clean your teeth and mouth area effectively. Give you a better gum health!

Inserting and removing nasal/Oral nozzle:

1. To insert the nasal/oral nozzle into the nozzle socket: insert it securely until you hear click sounds (FIG. 1).
2. To remove the nasal/oral nozzle from the device: pull it out to remove it (FIG. 2).

Recommended Technical:

For nasal cavity irrigating:

Fill the tank with lukewarm salty water and insert the nozzle with silicone head. Insert the nozzle into the nostril you want to clean. Activate the device by pressing the button several times and let the warm salty water come out from the other nostril.

For teeth and mouth area irrigating:

Fill the tank with lukewarm water and insert the nozzle without silicone head.

Hold the device in an upright position with the oral nozzle towards your teeth. Keep the device upright during use for best results. Keep your mouth slightly opened so that the water can flow out.

Press 'Power switch' to start irrigating.

Directing the water jet flow at a 90° degree angle to the teeth and gum.

Directing the water jet slowly along the teeth.

Directing the water jet along the gum line with nozzle tip at a 90° degree angle to the gum tissue.

Cleaning between the teeth and around any braces, crowns or bridgework thoroughly.

Do not spray water directly onto periodontal pockets.

Warning:

If you have had nasal cavity surgery, oral or dental surgery in the previous 2 months, consult your surgeon or dentist before using this appliance.

Do not jet the water to the tongue directly as well as to the ear, nose or other sensitive areas. The high pressure maybe will cause serious damage.

Children should be supervised and do not play with this devise.

Only fill with pure water or solutions recommend by dentist when you clean the oral area and teeth.

Only fill with warm salt water when you wash the nasal cavity. Some antibacterial solutions could shorten the use of the product.

People who have experience of nasal diseases like rhinitis or people have experience of gum pain may not be able to use this device and should consult the doctor or dentist.

Do not put the nozzle against your teeth or gum tightly; this may injure your gums.

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Opasnost:

If you have had surgery on your nose, teeth, mouth or jaw, contact your doctor or dentist before using this device.

Do not direct the water stream directly at the tongue, other sensitive parts of the body or the ears, as there is a risk of injury due to high pressure.

This product is not a toy, children should be supervised when using the device.

When cleaning the mouth and teeth, use only clean water or recommended solutions. Use warm water for the nasal cavity.

People with nasal conditions, such as inflammation of the nose or other, and people with infections of the mouth should consult a doctor before using the device.

Do not press the nozzle against the teeth or gums, as this can lead to injury.

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